


80X80mm

bluetooth digital hearing aids

Quick Start Manual

T010

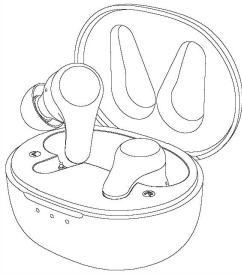

Please read this user manual thoroughly before using the product.

INTENDED USE

1. Medical purpose

- The hearing aid intends to compensate for impaired hearing.


2. Intended patients and environment

- The hearing aid uses for indoor and outdoor environment, and patients under 18 years old must require the doctor's approval.

3. Side effects

- For patients with sensitive skin, the silicone material may irritate your ear canals
- First use may cause slight pain to the ear canal due to maladjustment.

PRODUCT PROFILE

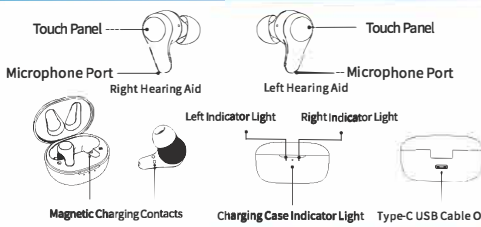

IN THE PACKAGE

Name	Quantity	Name	Quantity
User Manual	1	In-Ear Tip L	2
Hearing aids	2	In-Ear Tip S	2
Charging Case	1	Charging Cable	1


PERFORMANCE PARAMETERS

Product Model	T010
Output OSPL90 Max	115dB
FG50 MAX Gain	35±3dB
EQ Input Noise	30dB±3dB
Frequency Range(Not narrower than)	250Hz-6500 Hz
Working Current	8mA
Total Harmonic Distortion	5%
Bluetooth	BT5.0
Music Model	SBC AAC
Hearing Aid Battery Level	3.7V/50 mAh
Charging Case Battery Level	3.7V/400mAh
Bluetooth Distance	>10M
Play Time	>6 Hours
Comprehensive Use Time	>18Hours
Standby Time	168 Hours
Working Voltage	DC3.7V
Charging Port	Type-C
Charging In-put	DC 5V/1A

Note: The above parameter index testing is on basis of FONIX8000 Hearing Aid Test System; inspection standard: IEC 60118-7:2005, 2CC coupling cavity SB-1.

CHARGING HEARING AMPLIFIERS

PUT the devices into the charging case, close the lid, connect the cable and charge for 2.7 hours. Your devices can be charged at any time by returning them to the portable charging case, the charging case can hold 2.5 times full charges and both hearing amplifiers can be fully recharged in around 2 hours.

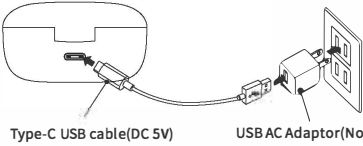

Type-C USB cable (DC 5V)      USB AC Adaptor (Not included)

Note: Please fully charge the hearing amplifier when using for the first time.

NOTES:

- When hearing amplifiers are charging, the two indicator lights will still on.
- When hearing amplifiers are fully charged, the two indicator lights will go off.
- When the power of the hearing amplifiers are low, pls charge will be broadcast every 2 minutes.
- When charging case is charging, the red indicator light will still on.
- When charging case is fully charged, the blue indicator light will still on.
- When you are not using the hearing amplifier, please put them back into the charging case to charge.
- For the first time using, we recommend that fully charge your hearing amplifier first.

WEARING YOUR HEARING AMPLIFIERS

① Remove the devices from the case and grasp it from the sides.

② Pinch the urethane ear tip with your fingers so it shrinks, it will expand once fit into your ear, creating a tight fit.

③ Insert the devices into your ear canal, gently twisting and rotating them to ensure a deep and secure fit.

Note: ① Hearing Amplifiers have a power on delay function, it will start after 8-10 seconds; ② When Hearing Amplifiers turn on, you may hear a beeping sound. The beeping sound will stop once it is correctly inserted into the ear.


Note: The Hearing Amplifiers are installed on M model ear tips, please choose the right ear tips to use according to your need.

Remark: ① Ensure the Hearing Amplifier has power of full power, ② Ensure the ear tips are firmly fixed, ③ Ensure the Hearing Amplifier turns on; ④ If the Hearing Amplifiers fail to turn on, please put them back into the charging case and cover the lid to charge, and repeat the first step again.

HEARING AMPLIFIER VOLUME CONTROL

Right Hearing Aid


Left Hearing Aid

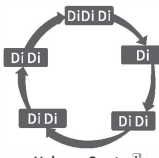

Note: ① You can touch the left and right Hearing Amplifier to control the volume of them, ② The volume of the two Hearing Amplifiers are independent adjustment, please adjust each one separately when using; ③ The Hearing Amplifiers have a memory function, it will automatically return to the volume you used before when using it next time; ④ Factory default for Hearing Amplifiers are Maximum Volume.

PAIRING BLUETOOTH

Open the charging case, the device will automatically enter into Bluetooth pairing mode, select "H010" from the Bluetooth list of your device and click to connect.

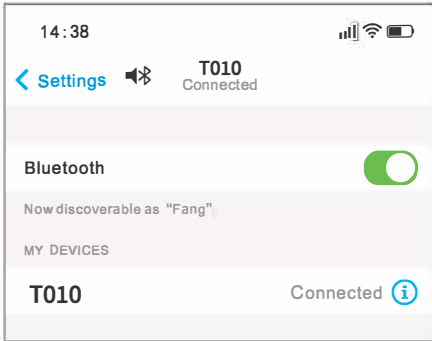

PHONE CALL

Right Hearing Aid

Left Hearing Aid

Answer the call

In the incoming call state, short press the L or R touch panel for 1 second to connect the incoming call.

Hang up the Call

In the call state, long press the L or R touch panel for 2 seconds to hang up the call.

Reject the call



In the incoming call state, long press the L or R touch panel for 2 seconds to reject the incoming call.

Note:



- ① Calling volume needs to be manually adjusted by phone volume,
- ② Music playing function needs to be manually adjusted by phone,
- ③ The volume of music playing is also amplified according to the hearing amplifier standard, Please follow the normal listening volume to adjust to your own volume, too much volume will further damage your hearing.

How to download the APP

1. APP can be installed on Android phones by scanning the following QR code with the following icon or search "amplifier" from Google Play.




2. APP can be installed on Apple phones by scanning the following QR code with the following icon or search "BT -Amplifier" from the Apple Store.




How to use the APP

Note: APP will bring its own memory function after being used by the hearing aid.


①

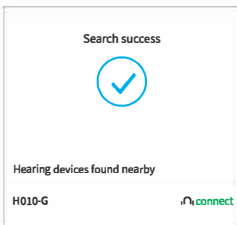


②

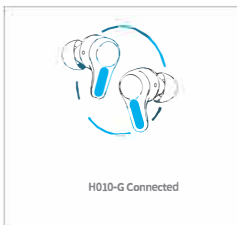

③

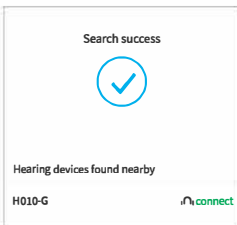

④

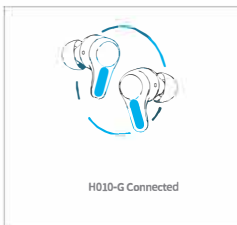


2. Use the APP

(1) Mode Selection:


This device has Outdoor mode, Indoor mode, Normal mode can be chose, you can choose the right mode according to your environment as below shown in (Figure 1).

(2) Volume Control:


The Volume of the Left or Right Hearing Amplifier can be adjusted by sliding up and down the knob, you can adjust the most suitable volume according to your hearing condition as below shown in (Figure 2).

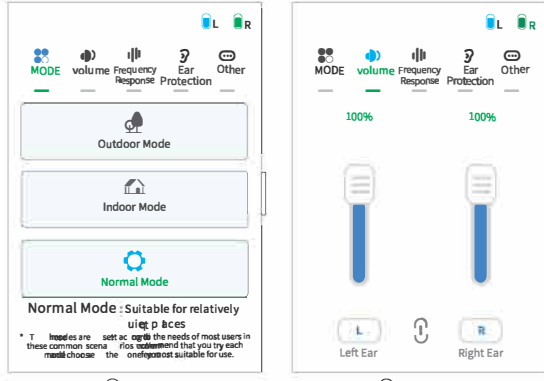

①

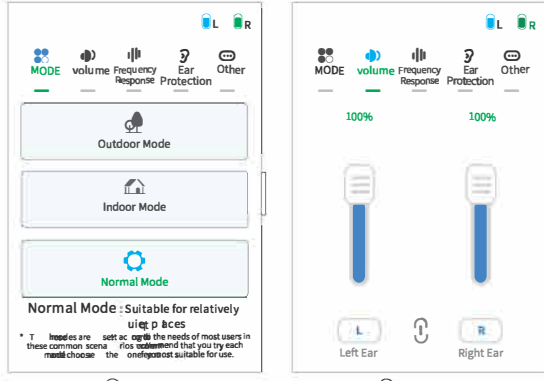

②

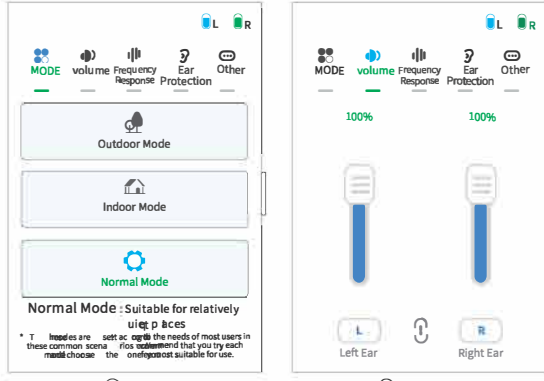


(3) Frequency Response Adjustment:


The Frequency Response of the Left or Right Hearing Amplifier can be adjusted by sliding up and down the knob to achieve the best hearing aids effect as below shown in (Figure 1).

(4) Ear Protection:


This APP comes with hearing protection function, excessive external or harsh noise can be compressed by sliding the bar switch to the left or right to achieve the best hearing aids effect as below shown in (Figure 2).

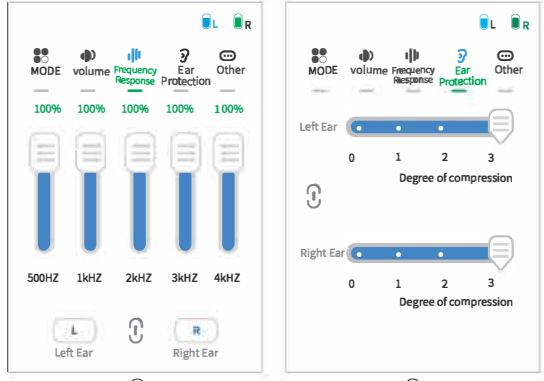

①

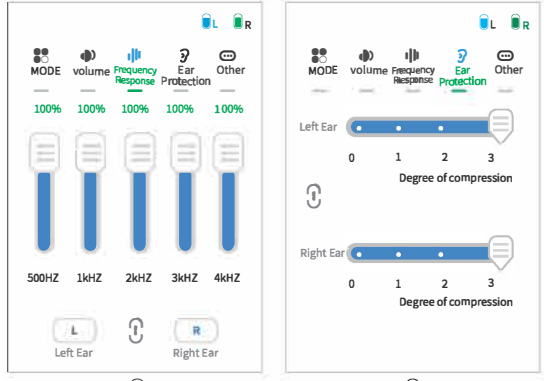

②

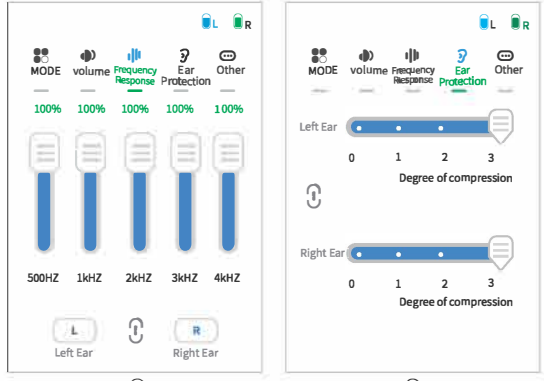

IMPORTANT NOTICE FOR PROSPECTIVE HEARING AID USERS

Important Safety Precautions

The following section contains important information regarding safe use. Please read the following carefully before using

1. Never attempt to disassemble, remodel or repair the product yourself. (There is a possibility of fire and/or injury.)

2. Do not use an unnecessarily loud volume.

Use at a high volume for extended periods of time may cause hearing loss or exacerbate existing hearing loss.

3. Do not expose this product to water, liquids, or moisture.

Do not use wet hands to handle the product. Contact with liquids may result in electrical shock, failure and injury.

4. Do not attempt to use outlets or outlet strips that would exceed the safe, rated value. Overloading an electrical circuit can result in fires.

5. Do not hit, crush, or use excessive force with the product. Also, do not handle the parts roughly, such as forcibly pulling them.

Rough and improper use can cause damage, electric shock, fire, and accidents.

6. If you experience or notice any of the following, stop using the product immediately, ignoring the following and continuing to use the product may cause unexpected accidents and injuries.

The product is broken or degraded. The product is damaged by chemicals. The product is deformed by heat.

7. Use of the product in the following ways can cause unexpected accidents, injuries, or physical damage. Do not allow children or people who require supervision to use.

8. Furthermore, the product contains small pieces which should be kept out of reach of children due to choking hazards. Do not lend or borrow this product. Do not continue to use if broken or damaged. Do not use while sleeping.

NORMALIZED SYMBOLS

	Consult instructions for using
	B type applied part
	Disposal: Please dispose of the device under the directive 2002/96/EC - WEEE (Waste Electrical and Electronic Equipment) and Don't dispose with domestic waste.
	Warning/Danger: Improper use might cause serious injuries.
	IP code of the device: this device's grade of against ingress of solid foreign objects - the access point of 2.5 mm (0.1 inch) - not penetrate; the grade of waterproof is vertically falling drops of water have no harmful effects.

FCC STATEMENT :

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

(1) This device may not cause harmful interference, and

(2) This device must accept any interference received, including interference that may cause undesired operation.

Warning: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

Reorient or relocate the receiving antenna.

Increase the separation between the equipment and receiver.

Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.

Consult the dealer or an experienced radio/TV technician for help