

sleepisol User Manual



Deep Sleep

Deep Sleep effect when using minimum twice a day(for 30 min) regularly



Concentration

Improving effect for Insomnia, Depression and Concentration from stress




Harmless to human body

Sleepisol is harmless to human body even in a longer use
Device provides(transmit) electrical impulses equal to the human current flow

How to Wear

- Make sure the sensors are on the temples and forehead on the face as image shown. Wear it on the comfortable spot of your face like a headband
- Please wear it comfortably on your face after cleansing or basic makeup

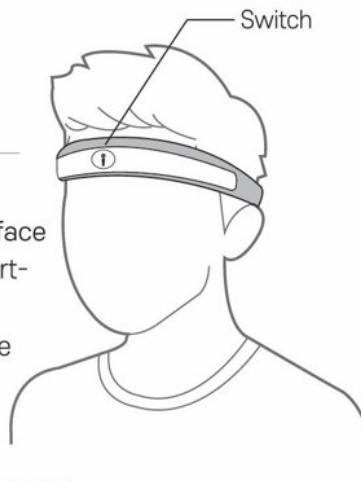

Power / Movement Button

- ① When the switch is touched for more than 1 second, the power On(green) / Off(orange) is shown with the LED & the alarm.
- ② When On(green) / Off(orange) is activated, the LED lights up 5 times.
- ③ When the power is turned on, the blue LED blinks every 3 seconds, works automatically for 30 minutes, then turns off after an alarm.

Charging Mode(C Type Connector)


- ① LED will flash when charging cable is connected
- ② Orange LED will continuously flash when charging is needed
- ③ Orange LED will appear while charging and Green LED will appear once it is fully charged
- ④ Please charge once for 1~2 hours

How to Use

sleepisol


1. Charge your device first

- Please Connect charging cable(C Type) to the charging connector for device charging
- Orange LED will appear while charging, Green LED will appear once charging is completed
- Please charge once for 1~2 hours

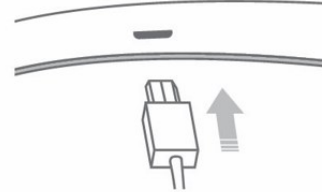

2. Turn on the power of Sleepisol

- Press the switch at the center of the device
- Green LED will flash 5 times once the power is turned on
- Once the power is on, Sleepisol will run for 30 mins and turned off automatically

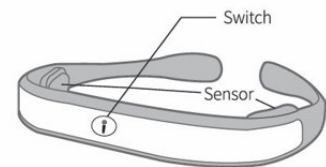

3. Wear the Sleepisol on your forehead


- Please wear the device as image shown, make sure the sensors are on the temples and forehead on the face.
- Please wear the device on your face comfortably after cleansing like a headband
- The power will automatically turned off (Red LED flash 5 times) after using the device for 30 mins



Application Manual

sleepisol

- App enables selection for various modes and operating environment
- Sleepisol automatically runs on Deep Sleep Mode without using App
- App is available on Android version 6.0 or higher & IOS version 11 or higher
- Please turn on the gps(location service) for user of android version

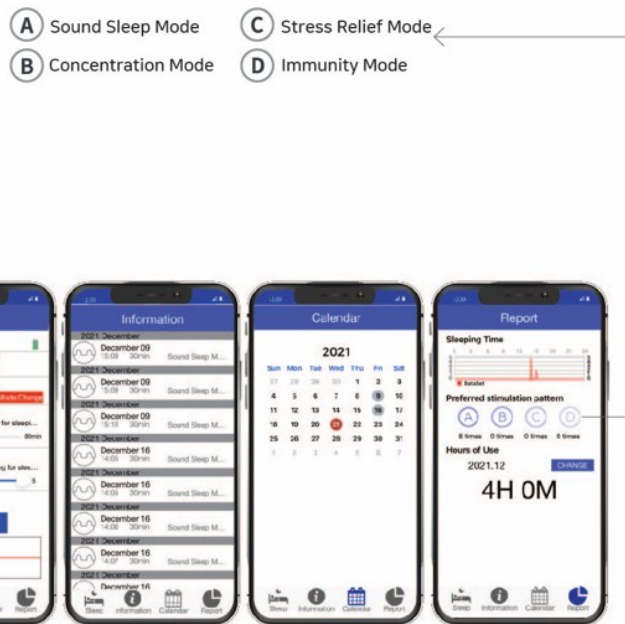

Important Safety Instruction

sleepisol

Please read all instructions before using 'Sleepisol'

- Please use this product for an appropriate time according to the instruction manual.
 - After using this product, slight afterimage may appear due to microcurrent, but it harmless to human body.
 - If there is skin irritation on the contact area after using this product, consult a specialist before using it.
 - Please do not use a fast charger which may cause product failure.
 - This product is vulnerable to water, please ensure the device is away from moisture.
 - Please ensure to keep device out of the reach of children.
 - Please ensure the product is not dropped or applied strong impact.
 - 'Sleepisol' is not responsible for any of disassemble, modification or unauthorized repair.
-

[Caution] Risk of fire or explosion if the battery is disposal into fire or hot oven, mechanically crushing or cutting

[Caution] Risk of explosion or the leakage of flammable liquid or gas if the battery is leaving in an extremely high temperature surrounding environment.

[Caution] Risk of explosion or the leakage of flammable liquid or gas if the battery is subjected to extremely low air pressure.

Specifications

Product Name	sleepisol
Model Name	HB-1
Description	A healthcare device that helps brain health by transmitting microcurrents to the brain.
Intended Use	Wellness or healthcare device (non-medical device)
Power	Power: 5.0Vdc, 35mA Battery: 3.7V, 38mAh, Li-Poly
RF Spec	Bluetooth LE 1M (2402 ~ 2480 MHz) Max Power (E.I.R.P): Max 0.98 dBm (E.I.R.P)
Operation Conditions	Temperature: +5°C to +40°C Relative Humidity: 15% to 93% Atmospheric Pressure: 700~1060 hPa
Storage and transportation conditions	Temperature: -25°C to +70°C Relative Humidity: Less than 93% Atmospheric Pressure: 700~1060 hPa
Warranty period	1 year
Customer service	Tel) +82-2-318-8890, Fax) +82-2-318-8842 E-mail) info@leesolbrain.com

Requirements

CE

This device can be operated in at least one Member state without infringing applicable requirements on the use of radio spectrum.

FCC

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

Reorient or relocate the receiving antenna.

Increase the separation between the equipment and receiver.

Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.

Consult the dealer or an experienced radio/TV technician for help.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device complies with RF exposure requirement.

Warranty Card

Product name	sleepisol
Date of Purchase	. . .
Where to buy	
Customer Information	Name
Address	
Contact no.	() -
Warranty Period	1 year from the date of purchase (excluding charging cables)

- After repair, the remaining period of the original warranty period is applied, including when exchanged
- For prompt A/S, fill out the form in the warranty, Please keep it
- In case of a product malfunction, please contact the retailer or customer center

Produced by Leesol Co., Ltd. (<http://leesolbrain.com>)

Tel: 82) 02-318-8890 , Email: info@leesolbrain.com

Operation hour ; Weekdays am10:00 ~ pm4:00