beurer

AS81

ENGLISH

Activity sensor

Instructions for use 3

ESPAÑOL

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Manual de instrucciones 18

FRANÇAIS

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Mode d'emploi 35

Manufactured for/Manufacturado por/

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Beurer North America LP

900 N Federal Highway, Suite 300

Hallandale Beach, FL 33009

www.beurer.com

Questions or comments? Call toll free

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**READ THIS MANUAL COMPLETELY AND CAREFULLY
BEFORE USING THIS PRODUCT**

Keep this manual in a safe location for future reference

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ENGLISH

CONTENT

1. IMPORTANT SAFETY INSTRUCTIONS.....	5
2. Introduction	7
3. Package Contents	8
4. To Use	8
5. Care and Maintenance	10
6. Technical Specifications	10
7. FCC Compliance Statement	11
8. Warranty.....	13

SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact Beurer customer service. Please see warranty for service contact.

Dear Customer,

Thank you for choosing this Beurer product. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air. Please read these instructions completely and carefully before using this product and keep them for later reference. Be sure to make them accessible to other users and observe the information they contain.

With best regards,
Your Beurer Team

1. IMPORTANT SAFETY INSTRUCTIONS

Signs and symbols

The following signs appear in the Safety Section (page 6) and in this manual on page 8.

READ AND UNDERSTAND THIS ENTIRE MANUAL INCLUDING THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.

**WARNING**

WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury.

**CAUTION**

CAUTION indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.

NOTICE

NOTICE addresses practices not related to personal injury, such as product and/or property damage.

2. Introduction

This activity sensor works with our 'beurer HealthCoach' app.

After the device is set up, it continuously records your physical activity and converts this activity into steps. Actual distance covered may differ from calculated values.

System requirements for the 'beurer HealthCoach' app

Bluetooth® 4.0, iOS from version 8.0, Android™ from version 4.4 with *Bluetooth*® low energy technology.

List of compatible devices:

The *Bluetooth*® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Beurer North America is under license. Other trademarks and trade names are those of their respective owners.

3. Package Contents

- 1 x Activity sensor AS81
- 1 x USB charging cable
- 1 x Instruction manual
- 1 x Quick start guide

4. To Use

NOTICE Do not use while swimming or while in the shower. This device is splash-proof, not waterproof.

1. Activate the device by pressing and holding the power button.
2. Charge the device fully before using. To charge, connect the USB charging cable to the device and to a powered USB port. Charging time is about three hours.

NOTE: The included cable is for charging only and cannot be used for data transfer.

3. Download and install the free 'beurer HealthCoach' app from your device's app store.
 4. When the device is charged, activate *Bluetooth*[®] on your mobile device. Start the app and follow the instructions.
 5. Select "AS81" from the list of devices within the app. You can set the activity target, sleep target, stride length, and your body weight. You can also set an alarm if desired.
 6. To sync with your mobile device, press and hold the power button for five seconds. Now get active!
 7. To transfer data to your mobile device, press and hold the power button for five seconds in all modes except "SLEEP" mode.
 8. To access Sleep Mode, press the power button repeatedly until "SLEEP" appears on the LCD Display. Press and hold the power button for three seconds to activate or deactivate, as applicable.
-  The 'beurer HealthCoach' app must be active to allow data transfer.

5. Care and Maintenance

- Clean the device with a slightly moistened cloth. Do not use detergents or solvents.
- Never immerse the device in water or other liquids.
- Keep the device away from lotions and creams, such as sunscreen.
- While charging, do not cover the unit. Only charge the unit in a dry environment at normal room temperature.

6. Technical Specifications

Model:	AS81
Type:	PR148
Product measurements (LxWxH):	9.9 in x 0.8 in x 0.5 in (252 x 19.6 x 11.5 mm)
Product weight:	approx. 0.6 oz (16 g)
Memory capacity:	30 days and 7 nights
IPX classification:	IPX 4
Transfer:	via <i>Bluetooth</i> [®] low energy technology

List of supported smartphones:

7. FCC Compliance Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference

in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

8. Warranty

Limited Lifetime Warranty For Original Purchaser

Your Beurer activity sensor, Model AS81, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer activity sensor, Model AS81, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at **1-800-536-0366** or at **info@beurer.com** to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty,

you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; old worn batteries; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of

Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE

WRITTEN WARRANTY. Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER. Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in

connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

For more information regarding our product line in the USA, please visit:
www.beurer.com

Manufactured for:
Beurer North America, LP
900 N Federal Hwy, Ste 300
Hallandale Beach, FL 33009 USA

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The *Bluetooth*[®] word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Beurer North America is under license. Other trademarks and trade names are those of their respective owners.