



## TABLE OF CONTENTS

<b>Overview</b> .....	<b>02</b>
General precautions	
General warnings	
Indications for use	
Usage & operating life	
Product parts	
Product overview	
 <b>Usage Instructions</b> .....	 <b>08</b>
1. Getting started: Disassembly	
2. Getting started: Before first use	
3. Assembly	
4. Putting on the pump	
5. Pumping	
6. Transferring milk	
7. Cleaning	
8. Charging	
9. Troubleshooting	
 <b>Technical Information</b> .....	 <b>23</b>

# OVERVIEW

---

## GENERAL PRECAUTIONS

1. If package appears damaged or tampered with, do not use and contact Customer Care.
2. If pumping is uncomfortable, change suction, turn off and reposition pump or remove your pump and discontinue session. If discomfort continues, reach out to a lactation consultant or healthcare professional.
3. Before each use, visually inspect each part for signs of wear and tear. If damage is found, discontinue use until parts have been replaced.

## GENERAL WARNINGS

1. Do not operate where aerosol spray products are used or oxygen is being administered.
2. Use product only for its intended use as described herein.
3. This is a personal care product for a single user and should not be shared between users.
4. Only use Willow Go™ pump parts. Do not substitute or modify pump parts.
5. Pump is not a serviceable device.
6. The product contains parts, such as the USB Charging Cable, which may present a hazard. Keep out of reach of children.
7. Never operate product if USB charging cable is damaged or not working properly.



- 
8. Do not attempt to open or repair pump. No modification of this equipment is allowed. The pump contains an internal non-replaceable lithium-ion battery pack.
  9. Do not operate if pump is damaged or immersed in water.
  10. Pump and parts are not provided sterile.
  11. If you are a mother who is infected with Hepatitis B, Hepatitis C or Human Immunodeficiency Virus (HIV), pumping breast milk will not reduce or remove the risk of transmitting the virus to your baby through your breast milk.
  12. Containers are marked with a 5 ounce (150 milliliter) maximum fill level. Monitor milk level during pumping to ensure overflow does not occur.

## INDICATIONS FOR USE

The Willow Go™ Breast Pump is intended to express milk from lactating women in order to collect breast milk from their breasts. The device is intended for a single user.

CONTRAINDICATION: No known contraindications.

## USAGE & OPERATING LIFE

Pump has an expected operating life of 12 months.

Pump diaphragm, duckbill, container bowl, container back and flanges are reusable parts and should be cleaned after each use. The expected operating life of these components is 3 months.

## PRODUCT DESCRIPTION

The Willow Go™ Breast Pump is a personal use, wearable breast pump for single and double pumping. The product contains two (2) of each of the following parts and accessories listed here: pump, pump diaphragm, container duckbill valve, 5 ounce (150 milliliter) container bowl, container back, 21 mm flange, 24 mm flange, and USB charging cable. Parts shown on page. All parts are provided non-sterile and should be cleaned prior to use.

Additional spare parts, including 7 ounce containers, 27 mm flanges, and sizing inserts are available for purchase at [onewillow.com](https://onewillow.com).



**USB Charging Cable**

*\*Wall charger sold separately*

## PRODUCT PARTS



**Flange**  
(21 mm & 24 mm)

**Pump**





**Pump Diaphragm**



**Container Back**




**Container  
Duckbill Valve**



**Container Bowl**  
5 ounce (150 milliliter)

## PRODUCT OVERVIEW

### NON-WASHABLE PARTS

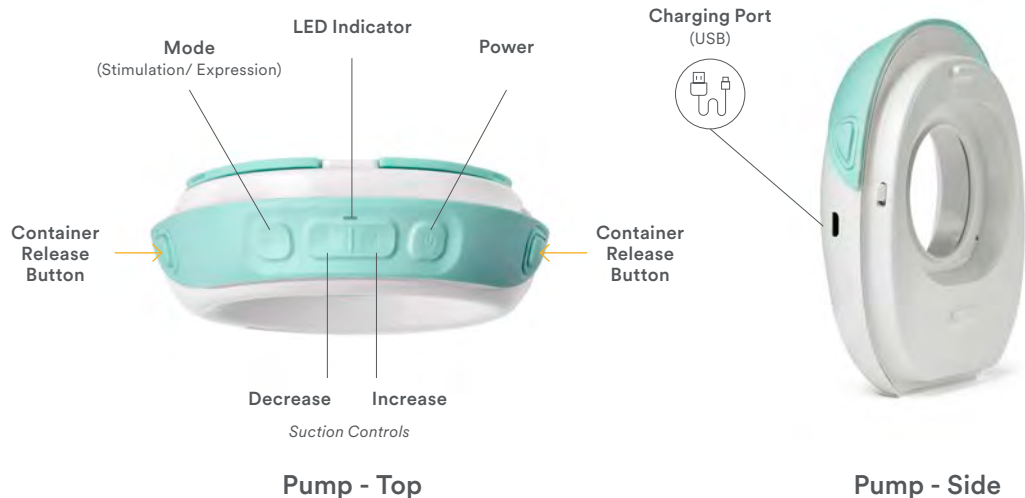

## WASHABLE PARTS

Pour Spout/Vent Hole

Volume Markers



Container Bowl




Container  
Duckbill Valve



Container Back



Pump Diaphragm

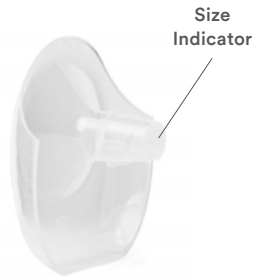

Size  
Indicator

Flange

# USAGE INSTRUCTIONS

## 1. GETTING STARTED: DISASSEMBLY

To get started, disassemble the pump.

1




Remove flange

2




Remove container



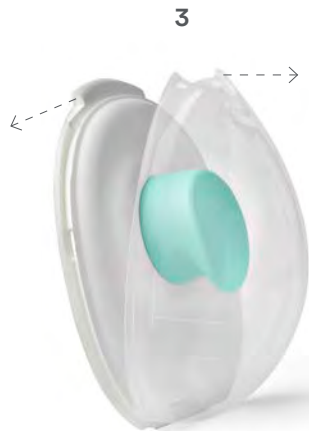

Remove  
container back

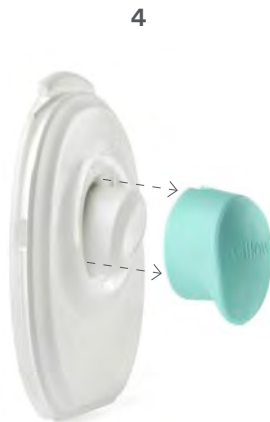

Remove duckbill



Remove diaphragm

---

## 2. GETTING STARTED: BEFORE FIRST USE

### BEFORE FIRST USE

#### Check your flange fit

Use the sizing guide included in the box to measure your nipple and find your flange size. Proper fit leads to greater comfort. 21 mm and 24 mm included. Purchase 27 mm flanges or sizing inserts at [onewillow.com](http://onewillow.com).



#### Boil parts

Boil the 5 washable parts for 5 minutes.

