



---

## Digital Blood Pressure Monitor

---

**Instruction Manual**

**Automatic Upper Arm Style  
Model No: BP100A**

## Table Of Contents

---

<b>TABLE OF CONTENTS.....</b>	<b>2</b>
<b>1. INTRODUCTION.....</b>	<b>4</b>
1.1. FEATURES OF THE BP100A.....	4
1.2. IMPORTANT INFORMATION ABOUT SELF-MEASUREMENT .....	4
<b>2. IMPORTANT INFORMATION ON THE SUBJECT OF BLOOD-PRESSURE AND ITS MEASUREMENT.....</b>	<b>5</b>
2.1. HOW DOES HIGH/LOW BLOOD-PRESSURE ARISE?.....	5
2.2. WHICH VALUES ARE NORMAL?.....	6
<b>3. THE VARIOUS COMPONENTS OF THE BLOOD-PRESSURE MONITOR.....</b>	<b>7</b>
<b>4. PUTTING THE BLOOD-PRESSURE MONITOR INTO OPERATION.....</b>	<b>8</b>
4.1. INSERTING THE BATTERIES .....	9
4.2. READING THE SET DATE .....	9
4.3. LANGUAGE TALKING SELECTION , USER SELECTION AND SETTING THE TIME / DATE.....	9
<b>5.CARRYING OUT A MEASUREMENT .....</b>	<b>11</b>
5.1. BEFORE THE MEASUREMENT .....	11
5.2. COMMON SOURCES OF ERROR .....	11
5.3. FITTING THE CUFF .....	12
5.4. MEASURING PROCEDURE .....	123
5.5. DISCONTINUING A MEASUREMENT .....	134
5.6. MEMORY – STORAGE AND RECALL OF THE MEASUREMENTS .....	134
5.7. MEMORY FULL .....	14
5.8. MEMORY– CANCELLATION OF ALL MEASUREMENTS ATTENTION! .....	14

5.9.BLUETOOTH FUNCTION.....	15
<b>6. APPEARANCE OF IHB(IRREGULAR HEART BEAT) DETECTION.....</b>	<b>15</b>
<b>7. ERROR MESSAGES /MALFUNCTIONS .....</b>	<b>17</b>
<b>8. CARE AND MAINTENANCE, RECALIBRATION.....</b>	<b>19</b>
<b>9. GUARANTEE.....</b>	<b>19</b>
<b>10. SERVICE LIFE .....</b>	<b>19</b>
<b>11. BATTERY LIFE : .....</b>	<b>19</b>
<b>12.SAFETY,CARE AND DISPOSAL.....</b>	<b>19</b>
<b>13.REFERENCE TO STANDARDS.....</b>	<b>21</b>
<b>14.REMARK:.....</b>	<b>21</b>
<b>15. TECHNICAL SPECIFICATIONS.....</b>	<b>22</b>
<b>16.MANUFACTURER'S DECLARATION.....</b>	<b>23</b>
<b>17.MANUFACTURER INFORMATION : .....</b>	<b>26</b>

## 1. Introduction

---

### 1.1. Features of the BP100A

---

The blood-pressure monitor BP100A (with integrated time/date display) is a fully automatic, digital blood-pressure measuring device for use on the arm, which enables very fast and reliable measurement of the systolic and diastolic blood-pressure as well as the pulse frequency by way of the oscillometric method of measuring.

The device offers a very high and clinical tested measurement accuracy and has been designed to provide a maximum of user-friendliness.

The device is intended for self-use in home .

Before using, please read through this instruction manual carefully and then keep it in a safe place.

For further questions on the subject of blood-pressure and its measurement, please contact your doctor.



### Attention!

#### 1.2. Important information about self-measurement

---

- Substitution of a different component might result in measurement error.
- cuff is replaceable only by an original .
- Do not use with neonatal patients.
- Do not intend to use with pregnant or pre-eclamptic patients
- It will cause harmful injury to the patient or effect the blood pressure due to connection tubing kinking.
- Too frequent measurements can cause injury to the patient due to blood flow interference.
- The application of the cuff over a wound can cause further injury.
- The application of the cuff and its pressurization on any limb where intravascular access or therapy, or an

arteriovenous (A-V) shunt, is present because of temporary interference to blood flow and could result in injury to the patient.

- Do not let the cuff and its pressurization on the arm on the side of a mastectomy
- Pressurization of the cuff can temporarily cause loss of function of simultaneously used monitoring ME equipment on the same limb.
- The need to check that operation of the automated sphygmomanometer does not result in prolonged impairment of patient blood circulation.
- Not intended to be used together with HF surgical equipment.
- Do not forget: self-measurement means control, not diagnosis or treatment. Unusual values must always be discussed with your doctor. Under no circumstances should you alter the dosages of any drugs prescribed by your doctor.
- The pulse display is not suitable for checking the frequency of heart pacemakers!
- In cases of IHB (Irregular Heart Beat), measurements made with this instrument should only be evaluated after consultation with the doctor.

### **Electromagnetic interference**

The device contains sensitive electronic components (Microcomputer). Therefore, avoid strong electrical or electromagnetic fields in the direct vicinity of the device (e.g. mobile telephones, microwave cookers). These can lead to temporary impairment of the measuring accuracy.

## **2. Important information on the subject of blood-pressure and its measurement**

---

### **2.1. How does high/low blood-pressure arise?**

---

The level of blood-pressure is determined in a part of the brain, the so-called circulatory centre, and adapted to the respective situation by way of feedback via the nervous system. To adjust the blood-pressure, the strength and frequency of the heart (Pulse), as well as the width of circulatory blood vessels is altered. The latter is effected by way of fine muscles in the blood-vessel walls. The level of arterial blood-pressure changes

periodically during the heart activity: During the «blood ejection» (Systole) the value is maximal (systolic blood-pressure value), at the end of the heart's «rest period» (Diastole) minimal (diastolic blood-pressure value). The blood-pressure values must lie within certain normal ranges in order to prevent particular diseases.

## 2.2. Which values are normal?

---

Blood pressure is too high if at rest, the diastolic pressure is above 90 mmHg and/or the systolic blood-pressure is over 160 mmHg. In this case, please consult your doctor immediately. Long-term values at this level endanger your health due to the associated advancing damage to the blood vessels in your body. Should the systolic blood-pressure values lie between 140 mmHg and 160 mmHg and/or the diastolic blood-pressure values lie between 90 mmHg and 100 mmHg, likewise, please consult your doctor. Furthermore, regular self-checks will be necessary.

With blood-pressure values that are too low, i.e. systolic values under 100 mmHg and/or diastolic values under 60 mmHg, likewise, please consult your doctor. Even with normal blood-pressure values, a regular self-check with your blood-pressure monitor is recommended. In this way you can detect possible changes in your values early and react appropriately. If you are undergoing medical treatment to control your blood pressure, please keep a record of the level of your blood pressure by carrying out regular self-measurements at specific times of the day. Show these values to your doctor. **Never use the results of your measurements to alter independently the drug doses prescribed by your doctor.**

Table for classifying blood-pressure values (unit: mmHg) according to World Health Organization:

<b>Range &amp; broadcasting</b>	<b>Systolic Blood-pressure</b>	<b>Diastolic Blood-pressure</b>	<b>Measures</b>
Hypotension	lower than 100	lower than 60	Consult your doctor
optimal	between 100 and 120	between 60 and 80	Self-check
normal	between 120 and 130	between 80 and 85	Self-check
high to normal	between 130 and 140	between 85 and 90	Consult your doctor
slight hypertension	between 140 and 160	Between 90 and 100	Seek medical advice
medium hypertension	between 160 and 180	Between 100 and 110	Seek medical advice
strong hypertension	Higher than 180	Higher than 110	Urgently seek medical advice!

#### Further information

- If your values are mostly standard under resting conditions but exceptionally high under conditions of physical or psychological stress, it is possible that you are suffering from so-called «labile hypertension». Please consult your doctor if you suspect that this might be the case.
- Correctly measured diastolic blood-pressure values above 120mmHg require immediate medical treatment.

### 3. The various components of the blood-pressure monitor

---






#### 4. Putting the blood-pressure monitor into operation



#### 4.1. Inserting the batteries

---

- a) Insert the batteries (4 x size AA 1.5V), thereby observing the indicated polarity.
- b) If the battery warning  icon appears in the display, the batteries remain 20% power to warn user the batteries will be run out .
- c) If the battery warning  icon appears in the display, the batteries are empty and must be replaced by new ones

**Attention!** • After the battery warning  icon appears, the device is blocked until the batteries have been replaced.

- Please use «AA» Long-Life or Alkaline 1.5V Batteries. The use of 1.2V Accumulators is not recommended.
- If the blood-pressure monitor is left unused for long periods, please remove the batteries from the device.

#### 4.2. Reading the set date

---

Please press the TIME button and the date will be shown in the display.

#### 4.3. Language selection, User selection and setting the time / date


---

##### Talking Language selection: Bilingual of English and Hindi

There are two languages of English and Hindi selectable.

To press and hold "ON/OFF" button around 5 seconds, then enter to language selection. A default language is English, to press "memory button "sequentially to switch language to Hindi

and mute (voice off) and go back to English and so on ...Then press "ON/OFF" button to save your selection and exit.

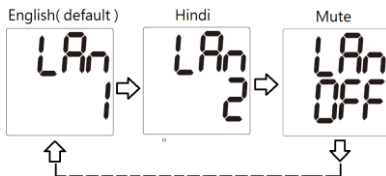

**User selection:** This advanced blood pressure monitor allows you to track blood pressure readings

for 2 individuals independently

- a) Before measurement, make sure you set the unit for the intended user. The unit can track results for 2 individuals. (User 1, User 2)
- b) Press the TIME button for at least 3 seconds. The display now indicates the set user, during which the set user blink, To confirm, press ON/OFF button
- c) Click the MEMORY button to select User
- d) We suggest the first person to take their pressure to be User 1.

### **Setting the time , date**

This blood-pressure monitor incorporates an integrated clock with date display. This has the advantage, that at each measurement procedure, not only the blood-pressure values are stored, but also the exact moment of the measurement. After new batteries have been inserted, the clock begins to run TIME 12:00 and DATE 1-01. You must then re-enter the date and current time. For this, please proceed as follows.

1. Press the TIME button for at least 3 seconds firstly , user icon will blink. Then press TIME button again the display now indicates the set year, during which the four characters blink.
2. The correct year can be entered by pressing the MEMORY button
3. Press the TIME button again. The display now switches to the current date, during which the first character (month) blinks.
4. The corresponding month can now be entered by pressing the MEMORY button.
5. Press the TIME button again. The last two characters (day) are now blinking
6. The corresponding day can now be entered by pressing the MEMORY button.
7. Press the TIME button again. The display now switches to the current time, during which the first character (Hour) blinks
8. The corresponding hour can now be entered by pressing the MEMORY button.
9. Press the TIME button again. The last two characters (Minutes) now blink.
10. The exact time can now be entered by pressing the MEMORY button
11. Press TIME button (or TIME / DATE or TIME): the unit of measurement will flash.

12. Press the "MEMORY to set the unit of measurement (mmHg or kPa)
13. Once you have made your settings, press the TIME button (or TIME / DATE or TIME). The setting is confirmed and the clock starts running.
14. Now after all settings have been made, press the TIME button once again. The date is briefly displayed and then the time. The input is now confirmed and the clock begins to run.

#### **Further Information**

With each press of the button (TIME, MEMORY) one input is made (e.g. switching over from hours to minutes mode, or altering the value by +1). However, if you keep the respective button depressed, you can switch more quickly to find the desired value respectively.

## **5. Carrying out a measurement**

---

### **5.1. Before the measurement**

---

- Avoid eating, smoking as well as all forms of exertion directly before the measurement. All these factors influence the measurement result. Try and find time to relax by sitting in an armchair in a quite atmosphere for about ten minutes before the measurement.
- Measure always on the same arm (normally left).
- Attempt to carry out the measurements regularly at the same time of day, since the blood-pressure changes during the course of the day.

### **5.2. Common sources of error**

---

**Note:** Comparable blood-pressure measurements always require the same conditions! These are normally always quiet conditions.

- All efforts by the patient to support the arm can increase the blood-pressure. Make sure you are in a comfortable, relaxed position and do not activate any of the muscles in the measurement arm during the measurement. Use a cushion for support if necessary.
- The performance of the automated sphygmomanometer can be affected by extremes of temperature, humidity and altitude.

- Avoid compression or restriction of the connection tubing.
- A loose cuff causes false measurement values.
- With repeated measurements, blood accumulates in the respective arm, which can lead to false results. Correctly executed blood-pressure measurements should therefore first be repeated after a 5 minute pause or after the arm has been held up in order to allow the accumulated blood to flow away (after at least 3 minutes).

### 5.3. Fitting the cuff

Insert air connector into air outlet shown in left photo and please make sure the fitting of the air connector completely and properly to avoid air leakage .

Cuff spec : arm size range 22~42cm

- a) The distance between the edge of cuff and the elbow should be approx. 2~3cm.



- b) Secure the cuff with the Velcro fastener, so that it lies comfortably and not too tight, whereby no space should remain between the cuff and the arm.

- c) Lay the arm on a table, with the palm upwards. Support the arm a little with a rest (cushion), so that the cuff rests at about the same height as the heart. Take care, that the cuff lies free. Remain so for 2 minutes sitting quietly, before beginning with the measurement.

- d) Let legs uncrossed, feet flat on the floor, back and arm supported.

### 5.4. Measuring procedure

After the cuff has been appropriately positioned, the measurement can begin:




a) Press the ON/OFF button, the pump begins to inflate the cuff. In the display, the increasing cuff-pressure is continually displayed.

b) After reaching the inflation pressure, the pump stops and the pressure slowly falls away. The cuff-pressure (large characters) is displayed during the measurement. When the device has detected the pulse, the heart symbol in the display begins to blink for every pulse beat. c) When the measurement has been concluded. The measured systolic and diastolic blood-pressure values as well as the pulse frequency are now displayed.

*Example (Fig.): Systole 118, Diastole 73, Pulse 75*

The measurement results are displayed, until you switch the device off. If no button is pressed for 3 minutes, the device switches automatically off, to save the batteries.

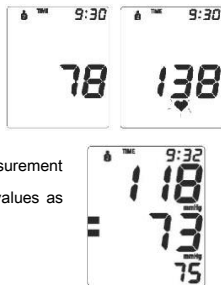

## 5.5. Discontinuing a measurement


---

If it is necessary to interrupt a blood pressure measurement for any reason (e.g. the patient feels unwell), the "ON/OFF" power button can be pressed at any time. The device then immediately lowers the cuff-pressure automatically.

## 5.6. Memory – storage and recall of the measurements

---


The blood-pressure monitor automatically stores each of the last 120 measurement values. By pressing the MEMORY button, an average value of the last 3 measurements as well as the last measurement and the further last 120 measurements (MR119,MR118,...,MR1)can be displayed one after the other

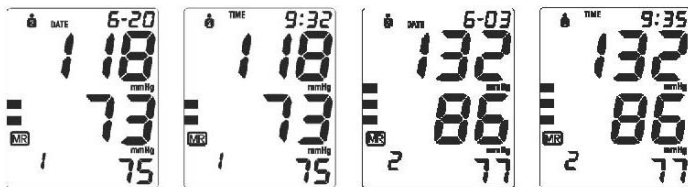

(MR1: Values of the last measurement) (MR2-MR120: Values of the measurement before MR1)

### 5.7. Memory full

Pay attention that the maximum memory capacity is not exceeded. When the memory is full, the old values are automatically Overwritten with new ones. When memory is full , the display shown 1 second as follows to remind you “ memory full “

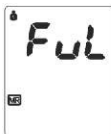

### 5.8. Memory– cancellation of all measurements Attention!

Before you delete all readings stored in the memory, make sure you will not need refer to the readings at a later date. Keeping a written record is prudent and may provide additional information for your doctor's visit. In order to delete all stored readings, depress the MEMORY button for at least 5 seconds, the display will show the symbol «CL» and then release the button. To permanently clear the memory, Press the MEMORY button while «CL» is flashing. To indicate deletion of stored readings.



### 5.9. Bluetooth function!

1. The product has bluetooth function, you can install the offered APP on your phone and to connect it on BPM device, then hit start to test. The test data will be transmitted to your phone for storage.

The operation method is as follows:

1. For the first time use: install the APP. After completion, the phone shows the app icon:



software icons:


2. When the installation is complete, set the user information. Including basic information, like name, gender, height, weight and so on. Enter the bluetooth connection interface and click "scanner";




(1)



(1.1)

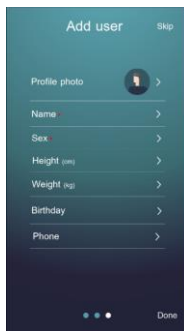

(2)

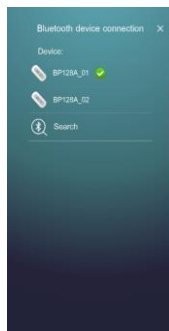

(3)



fig1: using interface

fig2: user information settings

fig3: Bluetooth connection interface

3. In the device standby state, Press / button. The LCD screen displays the "" symbol and flashes

4. After the device and mobile phone are successfully connected by bluetooth, the LCD screen of the device displays ", " and the mobile application switches to the test interface automatically (image4).

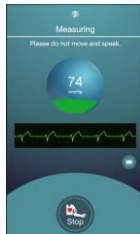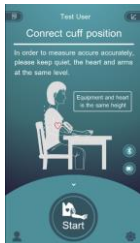

(4)  
(5)  
(6)  
(7)  
fig4:  
test

interface




fig5: measurement process

fig6: result of measurement


fig7: the measurement history

5, in the mobile application test interface, click "Start", the device began to measure blood pressure automatically (Figure 5), please do not move and speak.

6, complete the measurement, the application displays the measurement results automatically (image 6), the user can choose to save the data to the phone; (the measurement results will also be stored in the device)

7, if the user wants to look up the previous measurement history, then press the " " or " " button on the measurement interface, press the " " button to return to the measurement interface.

## 6. Appearance of IHB (Irregular Heart Beat) Indicator for early Detection

this symbol  indicates that certain IHB (irregular heart beat) were detected during the



measurement.

In this case, the result may deviate from your normal blood pressure – repeat the measurement. In most cases, this is no cause for concern. However, if the symbol appears on a regular basis (e.g. several times a week with measurements taken daily) we advise you to tell your doctor.

Please show your doctor the following explanation

#### **Information for the doctor on frequent appearance of the IHB (Irregular Heart Beat) indicator**

This instrument is an oscillometric blood pressure monitor that also analyses pulse frequency during measurement. The instrument is clinically tested. The IHB (Irregular Heart Beat) symbol is displayed after the measurement, if pulse irregularities occur during measurement. If the symbol appears more frequently (e.g. several times per week on measurements performed daily) we recommend the patient to seek medical advice.

The instrument does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage

## **7. Error messages /malfunctions**

---

If an error occurs during a measurement, the measurement is discontinued and a corresponding error code is displayed.

<b>Error No.</b>	<b>Possible cause(s)</b>
<b>ERR 1</b>	No pulse has been detected.
<b>ERR 2</b>	Unnatural pressure impulses influence the measurement result. Reason: The arm was moved during the Measurement (Artefact).
<b>ERR 3</b>	The inflation of the cuff takes too long. The cuff is not correctly seated.
<b>ERR 5</b>	The measured readings indicated an unacceptable difference between systolic and diastolic pressures. Take another reading following directions carefully. Contact you doctor if you continue to get unusual readings.
<b>Err8</b>	The pressure in the cuff is too high (over 290 mmHg)

**Further Information** - The level of blood-pressure is subject to fluctuations even with healthy people.

Important thereby is, that comparable measurements always require the same conditions (Quiet conditions)! If, in spite of observing all these factors, the fluctuations are larger than 15mmHg, and/or you hear irregular pulse tones on several occasions, please consult your doctor. For licensing, the device has been subjected to strict clinical tests, by which the computer program used to measure the blood-pressure values was tested by experienced specialist doctors in Germany. The same computer program is used in every individual device, and has thus also been clinically tested. The manufacture of the devices takes place according to the terms of the European standard for blood-pressure measuring devices (see technical data) you must consult your specialist dealer or chemist if there are technical problems with the blood-pressure instrument. Never attempt to repair the instrument yourself! Any unauthorised opening of the instrument invalidates all guarantee claims!

#### **Other possible malfunctions and their elimination**

If problems occur when using the device, the following points should be checked and if necessary, the corresponding measures are to be taken:

<b>Malfunction</b>	<b>Remedy</b>
The display remains empty when the instrument is switched on although the batteries are in place.	<ol style="list-style-type: none"> <li>1. Check batteries for correct polarity and if necessary insert correctly.</li> <li>2. If the display is unusual, re-insert batteries or exchange them.</li> </ol>
The device frequently fails to measure the blood pressure values, or the values measured are too low (too high).	<ol style="list-style-type: none"> <li>1. Check the positioning of the cuff.</li> <li>2. Measure the blood-pressure again in peace and quiet under observance of the details made under point 5.</li> </ol>
Every measurement produces a different value although the instrument functions normally and the values displayed are normal	<ol style="list-style-type: none"> <li>1. Please read the following information and the points listed under «Common sources of error». Repeat the measurement.</li> </ol> <p style="text-align: center;"><b>Please note: Blood pressure fluctuates continually so successive measurements will show some variability.</b></p>

<p>Blood pressure measured differs from those values measured by the doctor.</p>	<p>1. Record the daily development of the values and consult your doctor. <b>Please note: Individuals visiting their doctor frequently experience anxiety which can result in a higher reading at the doctor than obtained at home under resting conditions.</b></p>
--	--

## 8. Care And Maintenance, Recalibration

---

- a) Do not expose the device to either extreme temperatures, humidity, dust or direct sunlight.
- b) The cuff contains a sensitive air-tight bubble. Handle this carefully and avoid all types of straining through twisting or buckling.
- c) Clean the device with a soft, dry cloth. Do not use petrol, thinners or similar solvent. Spots on the cuff can be removed carefully with a damp cloth and soapsuds. The cuff must not be washed!
- d) Do not drop the instrument or treat it roughly in any way. Avoid strong vibrations.
- e) Never open the device! Otherwise the manufacturer calibration becomes invalid!

## 9. Guarantee

---

The blood-pressure monitor BP100A is guaranteed for 2 years from date of purchase. The guarantee does not apply to damage caused by improper handling, accidents, not following the operating instructions or alterations made to the instrument by third parties.

**The guarantee is only valid upon presentation of the guarantee card filled out by the dealer.**

## 109. Service life

---

5 years

## 11. Battery life:

---

1000 times measurement with 4- size "AA" alkaline Batteries

## 12. safety, care and disposal

---



## **Safety and protection**

- This instrument maybe used only for the purpose described in this booklet. The manufacturer cannot be held liable for the damage caused by incorrect application.
- This instrument comprise sensitive components and must be treated with caution. Observe the storage and operating condition described in the “Technical specifications” section !
- Protect it from water and moisture, extreme temperatures, impact and dropping, contamination and dust, direct sunlight, heat and cold
- The cuffs are sensitive and must be handled with care
- Only pump up the cuff once fitted
- Do not use the instrument close to strong electromagnetic fields such as mobile telephones or radio installations
- Do not use the instrument if you think it is damaged or notice anything unusual.
- If the instrument is not going to be used for a prolonged period the batteries should be removed.



- Read the additional safety instructions in the individual sections of this booklet. Ensure that children do not use the instrument unsupervised: some parts are small enough to be swallowed
- Must use the recognized accessories, detachable parts and materials, if the use of other parts or materials can degrade minimum safety
- A warning to remove primary batteries if the instruments is not likely to be used for some time

## **Instrument care**

Clean the instrument only with a soft, dry cloth

## **Disposal**



- Batteries and electronic instruments must be disposed of in accordance with the locally

applicable regulations, not with domestics waste.

### 13. Reference To Standards

Device standard: Device corresponds to the requirements of the European standard for non-invasive blood- pressure monitor

IEC60601-1-6:2010+A1:2013/ EN60601-1-6:2010+A1:2015
IEC60601-1:2015+A1:2012/EN60601-1:2006+A11:2011+A1:2013+A12:2014
IEC60601-1-2:2014/ EN60601-1-2:2015
IEC/EN60601-1-11:2015
IEC80601-2-30:2009+A1:2013/EN80601-2-30:2010+A1:2015

ISO81060-2

EN1060-4 – clinical investigation

ANSI / AAMI SP10, NIBP ,

IEC80601-2-30:2009 + corrigendum 2010





Electrical compatibility: Device fulfils the stipulations of the







IEC/EN 60601-1 ,

IEC/EN 60601-1-2

The stipulations of the EU-Guidelines 93/42/EEC for Medical Products Class IIa have been fulfilled.

### 14.Remark:

	Some electrical and electrical equipments forbid to abandon and disposal at will		TUV NO.
	Manufacturer's name and address		Reading Instruction Book before use

	Inapplicable baby		Type B equipment
	Cuff Connector		Keep Dry
	Attention consult accompanying documents		Wellkang Tech Consulting Suite B ,29 Harley Street, LONDON W1G 9QR ,United Kingdom

## 14. Technical specifications

<b>Measurement Procedure:</b>	Oscillometric , corresponding to Korotkoff method: Phase I : systolic , Phase V : diastolic
<b>Display:</b>	Digital display
<b>Measuring range:</b>	Pressure: 30 to 280 mmHg (in 1 mmHg increment) Pulse: 40 to 199 beat/minute
<b>Static accuracy:</b>	Pressure: $\pm 3\text{mmHg}$ / Pulse: $\pm 5\%$ of reading
<b>Measuring resolution :</b>	1mmHg
<b>Inflation:</b>	Automatic inflation by internal pump
<b>Memory function:</b>	2 x 120 memories for 2 users (SYS, DIA, Pulse)
<b>Decompression:</b>	Constant exhaust valve system
<b>Power source:</b>	4- size "AA" alkaline Batteries
<b>Operation temperature:</b>	5~40°C/41~104°F
<b>Operation humidity:</b>	15%~85%RH maximum
<b>Storage temperature:</b>	-10~55°C/14~131°F
<b>Storage humidity:</b>	10%~95%RH maximum

<b>Dimensions :</b>	136× 112× 71 ±1.0 mm
<b>Weight :</b>	510 g±5g (including batteries and cuff)
<b>Cuff pressure display range:</b>	0~299mmHg/0~39.9kPa
<b>Electrical shock protection:</b>	Internal power unit
<b>Safety classifications:</b>	Type B equipment
<b>Mode of operation:</b>	Continuous operation
<b>Protection against ingress of water:</b>	IP22
<b>Accessories:</b>	M-size Cuff , 4 "AA" batteries, instruction manual ,warranty card

Please be noticed the power adapter is not supplied from the origin ,users can buy the adapter in the market which must comply to EN60601-1,EN60601-1-2



## 15.Manufacturer's Declaration

The BP100A is intended for use in the electromagnetic environment specified below. The customer or the user of the BP100A should assure that it is used in such an environment.


### Electromagnetic Emissions: (IEC80601-1-2)

Emission Test	Compliance	Electromagnetic Environment
RF emission CISPR 11	Group 1	The BP100A uses RF energy only for internal functions. Therefore, this RF emission is extremely weak and there is little chance of it creating any kind of interference whatsoever with nearby electronic equipment.
RF emissions CISPR 11	Class B	The BP100A is suitable for use in all establishments, including domestic establishments and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/flicker IEC 61000-3-3	Not applicable	

**Electromagnetic Immunity: (IEC60601-1-2)**

<b>Immunity test</b>	<b>IEC60601-1-2 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment -guidance</b>
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electric fast transient/ burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV differential mode ±2 kV common mode	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % $U_T$ ( 95% dip in $U_T$ ) for 0.5 cycle 40 % $U_T$ ( 60% dip in $U_T$ ) for 5 cycles 70 % $U_T$ ( 30% dip in $U_T$ ) for 25 cycles <5 % $U_T$ ( 95% dip in $U_T$ ) for 5 sec.	Not applicable	Mains power quality should be that of a typical commercial or hospital environment. If the user of the upper arm stlye requires continued operation during power mains interruptions, it is recommended that the BP100A be powered from an uninterruptible power supply or a battery.
Power frequency (50/ 60 Hz) magnetic field IEC 61000-4-8	3 A/m	Not applicable	Not applicable
Note: $U_T$ is the a.c. mains voltage prior to application of the test level.			
<b>Immunity test</b>	<b>IEC60601-1-2 test level</b>	<b>IEC60601-1-2 test level</b>	<b>Electromagnetic environment - guidance</b>
			Portable and mobile RF communications equipment should be used no closer to any partof the BP100A,



Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz 80% AM (2Hz)	3 Vrms	including cables, than there commended separation distance calculated from the equation applicable to the frequency of the transmitter. <b>Recommend separation distance</b> 3V $d = 1.2 \times P^{1/2}$ 80MHz to 800 MHz $d = 2.3 \times P^{1/2}$ MHz to 2.5 GHz where $P$ is the maximum output power rating of the transmitter in watts (W) according to he transmitter manufacturer and $d$ is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters as determined by an electromagnetic site survey <sup>a</sup> , should be less than the compliance level in each frequency range <sup>b</sup> . Interference may occur in the vicinity of equipment marked with the following symbol:
Radiated RF IEC 61000-4-3	3 Vrms 80 MHz to 2.5 GHz 80% AM (2Hz)	3 V/m	

Note1: At 80 MHz and 800 MHz, the higher frequency range applies.

Note2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

- <sup>a</sup> Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless)telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the BP100A is used exceeds the applicable RF compliance level above, the BP100A should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the BP100A.
- <sup>b</sup> Over the frequency range 150 kHz to 80MHz, field strengths should be less than 3 V/m.

Recommended Separation Distances:

Recommended separation distance between portable and mobile RF communications equipment and

the BP100A

The BP100A is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the BP100A can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the BP100A as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1.2 \times p^{1/2}$	80 MHz to 800 MHz $d = 1.2 \times p^{1/2}$	800 MHz to 2.5 GHz $d = 2.3 \times p^{1/2}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in metres (m) can be determined using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note1: At 80MHz and 800MHz, the separation distance for the higher frequency range applies

Note2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

## 17.manufacturer information :



Shenzhen Combei Technology Co.,LTD

11-5B No.105, Huanguan South Road, Dahe Community, Guanlan, Longhua New District,

Shenzhen, 518110 Guangdong, China

manufacturer contact : Mr. Kevin Fong E-mail : kevin.fong@163.com

TEL:+86-755-29588956 29408890 29408891 EXT:818,809

## FCC Notice:

This device complies with Part 15 of the FCC rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference. (2) This device must accept any interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

**WARNING:** Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

### RF Exposure

The equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This device should be installed and operated with minimum distance 5mm between the radiator & your body.