Instructions For Use

Help and Support

The Sword Health app can be downloaded from the Apple App Store (for iPhone) or Google Play (for Android).

For assistance, please contact our Customer Care Team at help@swordhealth.com

Device Compatibility

Sword Bloom is provided with the Bloom Pod, which is designed to be used with a Bluetooth-enabled smartphone or tablet device capable of running the Sword Health app. Please check the Sword Health app for the current list of Android and iOS supported versions.

Intended Use

Sword Bloom is a system that includes software as a medical device (SaMD) and related accessories, including an intravaginal pelvic device, intended to guide members through an exercise program directed at pelvic floor muscles, monitor and collect information on the movement of the pelvic floor, and provide real-time feedback to the members on their performance.

The exercise program that members follow is monitored and adjusted by a pelvic health specialist based on the member's needs, and information collected by the device.

Intended Users

Sword Bloom is intended to be used by individuals with vaginal anatomy above the age of 18, to restore and improve movement, function and muscle strength of the pelvic floor. It can be used both by individuals who have been recommended a course of exercise by a suitably qualified professional with experience in pelvic physical therapy and also by individuals who have not been affected by specific injury or disability-related to pelvic floor issues.

Both the Sword Health App and the Bloom Pod are intended to be used by lay persons in the home (or similar) environment with sufficient room to perform the exercises. No specific education or training prior to using the device is required.

Indications for Use

Sword Bloom is primarily used to restore movement, function and muscle strength of the pelvic floor. It can also be used outside of its medical purpose by healthy individuals who have not been affected by injury, illness or disability for general health and wellbeing purposes, to work on improving the function of their pelvic floor.

What's Inside

Bloom Pod

A sensor used to receive feedback on your pelvic floor exercise performance.

Bloom Pod Case

A washable bag for storing your Bloom Pod.

Lubricant

A water-based lubricant to facilitate the Bloom Pod placement.

Pod Case Care Instructions

Hand wash cold

Do not bleach

Do not twist or wring

Reshape and lay flat to dry

Iron at medium temperature if necessary

Precautions

Only use Sword Bloom for its intended use as described in this manual. It should not be utilized as a treatment or preventive measure for any medical conditions.

If you have any doubts regarding its use, it is advised to consult with a licensed healthcare professional.

If the Bloom Pod package appears damaged, the tamper-proof shrink wrap is removed and/ or otherwise tampered with before use, do not use and contact our Customer Care Team at help@swordhealth.com.

This is a single-user product.

The use of this device by more than one person may present a health risk.

Do not use the Sword Bloom if:

- You have given birth within the last six weeks.
- You are in the first trimester of your pregnancy.
- You experience persistent pain when inserting or using the Bloom Pod.
- You have a medical condition affecting your pelvic health (for example, abnormal vaginal discharge or severe prolapse) and have not spoken to your Bloom pelvic health specialist or have not been cleared by a medical provider.

- You are allergic to medical-grade silicone.
- The Bloom Pod has been modified, damaged, and/or tampered with.

While there are no technical reasons not to use Sword Bloom during the second and third trimesters of pregnancy, we don't recommend using unless you have consulted with a licensed healthcare professional first.

Do not leave the Bloom Pod inside your vagina for more than half an hour at a time.

The Bloom Pod is made with medical-grade silicone. However, if you experience skin irritation, discontinue use immediately and consult a healthcare professional. For safe operation, please read and carefully follow the instructions in the "Clean the Bloom Pod" section.

During normal use, the temperature on the applied part of the Bloom Pod can reach up to 41,5°C.

It is recommended that you consult with a licensed healthcare professional before using the Bloom Pod if you have an active implanted device that would be affected by a magnetic field (e.g. IUD, pacemaker, implantable cardioverter defibrillator.)

Bloom Pod is not a substance-based device and the materials used are compliant with REACH, Prop65, and RoHS.

Use of the Bloom Pod adjacent to other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Use of accessories other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

The Bloom Pod uses Radio Frequency (RF) energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. The Bloom Pod is suitable for use in all establishments, including domestic establishments.

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Bloom Pod. Otherwise, degradation of the performance of this equipment could result.

Sword Bloom was tested against the emission requirements of IEC 60601-1-2, being classified as Class B, Group 1. It passed the testing for all of the following: Radiated RF Emissions; equipment Immunity testing for EN 61000-4-2 Electrostatic Discharges and the IEC 61000-4-3 Radiated RF EM Fields and IEC 61000-4-3 Proximity Wireless fields, IEC 61000-4-8 Power frequency magnetic field, IEC 61000-4-39 Radiated fields in close proximity immunity test with levels defined for home healthcare environment in accordance with IEC 60601-1-2 2015 and IEC 60601-1-2 AMD1_2020.

Contraindications

Sword Bloom is not designed to be used by members presenting with any of the following conditions:

- Symptoms/signs compatible with active pelvic infection.
- Symptoms/signs compatible with acute and serious underlying medical conditions (neurological, cardiac, respiratory, gastrointestinal, genitourinary, metabolic or other) precluding participation in an exercise program.
- Respiratory, cardiovascular or metabolic pathology where light to moderate physical exercise exertion is medically contra-indicated.
- Bone fractures or other musculoskeletal conditions in which mobilization of pelvic floor muscles or lower limbs is contraindicated.
- Aphasia, dementia or any other psychiatric comorbidity that interferes with the use of the medical device.
- Complicated or high-risk pregnancy
- Pelvic health complaints related to conditions outside of the program's coverage: i.e.- untreated, active pelvic inflammatory disease (PID).

In addition, the use of Sword Bloom is contraindicated in the presence of continuity solutions or other local complications affecting the vagina.

Using the Bloom Pod for the First Time

- Download the Sword Health app.
- Open the app and click to start a new session. Follow the steps provided by the app.
- When prompted by the app, press the power button until the blue light starts flashing, and it will automatically pair with your phone. When connected, the LED light will remain static.

Inserting the Bloom Pod

Clean the Bloom Pod

Remember to wash your hands and the Bloom Pod with warm, unscented soapy water before use, rinsing thoroughly.

Insert the Bloom Pod

The Bloom Pod is easy to insert. Use the grip at the start of the tail to make insertion easier. The pebble-shaped part goes inside your vagina, and the tail remains outside, facing forward. Ensure full insertion of the pebble-shaped part. If insertion feels uncomfortable, consider using the provided water-based lubricant.

How to Exercise

Let's start by lying down - On a comfortable surface, lie down on your back with your knees bent and slightly apart. We recommend putting a pillow behind your head for extra comfort.

To make sure you know this part of your body, let's quickly go over your pelvic floor. Your pelvic floor is a set of muscles, and like other muscles in your body, it can contract (tighten), relax (resting state), and lengthen (stretch). The pelvic floor needs to move in these directions for overall pelvic health.

Your pelvic floor has several openings to let you pee, poop, and perform other bodily functions and activities. The Bloom Pod will be inserted into the vagina for your exercises.

For pelvic contraction, imagine that you are trying to stop yourself from peeing. In contraction, the pelvic muscles are lifting inwards and upwards.

For pelvic lengthening, imagine that you are trying to start a flow of urine or pass gas. In lengthening, the pelvic muscles are dropping down and stretching away.

When exercising, remember to:

- Maintain a relaxed stomach, buttocks and thighs.
- Try not to hold your breath.
- Breathe in to lengthen your pelvic floor.
- Breathe out to contract your pelvic floor.

The Sword Health app provides more guidance during your sessions.

Removal of the Bloom Pod

At the end of the session remove the Bloom Pod. Use the grip at the start of the tail to pull the pod and make the removal easier. Then, wash your hands and the pod with warm water and unscented soap. Let the pod dry before returning to the case.

Taking Care of the Bloom Pod

Charging

No need to worry about that! The Bloom Pod is not chargeable. It includes a built-in lithium-ion battery that dictates the lifetime of the device: 24 hours of use due to battery degradation during expected use.

When the battery is running low, a warning will be displayed in the Sword Health app to contact the Customer Care Team.

Cleaning

Clean the Bloom Pod before and after every use with warm, unscented soapy water. Remember to rinse thoroughly. It is important to keep it completely dry before storing it.

Misuses

- Do not wash the Bloom Pod in boiling water.
- Do not sterilize.
- Do not use cleaning agents other than warm, unscented soapy water.
- Do not freeze or microwave the Bloom Pod, nor place it in the oven.
- Do not clean the Bloom Pod in a washing machine.
- Do not leave the Bloom Pod trapped under anything.
- Do not use the Bloom Pod in the bath.
- Do not share your Bloom Pod.
- Do not use it in any body region other than the vagina.

- Do not allow continuous contact of the Bloom Pod with the vaginal mucosa for more than 30 minutes.

Storing

- Keep the Bloom Pod clean, dry and snug in its carry case.
- Keep out of reach from children & animals.

Things to Avoid

- Extreme heat
- Extreme cold
- Sustained pressure
- Direct sunlight

Transport and Storage Conditions

-20°C/-4°F to 45°C/113°F, 10% to 85% RH, 70-106 kPa

Operating Conditions

10°C/50°F to 40°C/104°F, 10% to 85% RH, 70-106 kPa

Disposal

The Bloom Pod contains a Li-ion battery. Please clean the device prior to disposal. Please follow your state and local legislation for the recycling of electrical waste and electronic equipment.

For more information about where you can drop off your electrical and electronic waste for recycling, please contact your local authority.

Troubleshooting

If you need any help, please get in touch with our Customer Care Team at [**help@swordhealth.com**](mailto:help@swordhealth.com)

For all FAQs please navigate to the Sword Health app, the Bloom website (swordhealth.com/solutions/bloom) or contact [**help@swordhealth.com**](mailto:help@swordhealth.com)

Certifications

Bloom Pod (Model: BP01; FCC ID: 2AQ4A-BP1)

FCC statement

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful

interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the manufacturer or an experienced radio/TV technician for help.

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment.

Certification markings can be found under the tail of the Bloom Pod.

Unique Device Identifier (UDI)

The UDI is the unique device identifier which is a regulatory requirement and is displayed in full on the SaMD label (About page) for the mobile version and on the physical label for the system. The SaMD UDI-PI (8012) indicates the software version of Sword Bloom (version 7) with 'Y' denoting the minor software version and 'Z' denoting the patch. The system UDI-PI includes the (11) manufacturing date (YYMMDD) and the (21) serial number (SN) of the system.

SaMD UDI: (01)05065013876023(8012)7Y.Z

System UDI: (01)05065013876030(11)YYMMDD(21)SN

Instructions for Use Symbols

Consult Sword Bloom
Instructions for Use

Manufacturer

Date of Manufacture

Catalogue Number

Batch Code

Caution, there are specific warnings or precautions associated with the device which are not found on the label – consult the instructions for use (if applicable)

Medical Device

IPX7

Degree of Protection according to IEC 60529. IPX7: N1=X, N2=7, Protected against the effects of temporary immersion in water

Unique Device Identifier in both readable plain-text and Automatic Identification and Data Collection (AIDC) technology (2D-matrix) formats

BF applied part complying with IEC 60601-1

Temperature limitation for transport and storage

Humidity limitation for transport, storage, and operation

Atmospheric pressure limitation for transport, storage, and operation

Temperature limitation for operation

Liability

In no event will Sword Health S.A., or any of its affiliates, be liable regarding misuse of Sword Bloom, including negligence or uses not described in this document. Whilst being a device for restoring or improving movement, function and muscle strength of the pelvic floor, use of Sword Bloom does not automatically guarantee patient improvement. Every member using Sword Bloom must be under the care of a pelvic health professional.

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Sword Bloom

SB-IFU-US-02
Revised on 16 May 2024

© Sword Health S.A. All rights reserved.

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Sword Health Inc.