en

CADD®-Solis Communication Module

Model 2131

Operator's Manual

CADD®

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Table of Contents

Warnings	5
Cautions	6
About The CADD®-Solis Communication Module, Model 2131	6
Installing the CADD®-Solis Communication Module	7
Charging the Rechargeable Battery	11
Configuring Wireless Settings Using the CADD®-Solis Network Setup Utility Software	12
Turning Wireless Radio On/Off.	13
Viewing Wireless Status and Settings	13
Troubleshooting	14
Transportation	15
Disposal / Collect Separately	15
REACH Information	15
Part 15 Certification	15
Cleaning and Disinfecting the Communication Module	15
Specifications	17
Limited Warranty	19
French / Français	20
German / Deutsch	36
Italian / Italiano	52
Spanish / Español	68
Portuguese / Português	84
Dutch / Nederlands	100
Swedish / Svenska	116
Danish / Dansk	132
Japanese / 日本語	148

IMPORTANT: The rechargeable battery, in the CADD®-Solis Communication Module, must be internally connected and fully charged within the pump before placing the system into service. Review and follow the instructions in this Operator's Manual to install and charge the battery prior to use.

Read the instructions completely before installing the CADD®-Solis Communication Module to the CADD®-Solis Ambulatory Infusion Pump, Model 2110. Retain these instructions for future reference.

For detailed instructions, warnings, specifications, and additional information for operating the CADD®-Solis Ambulatory Infusion Pump, refer to the pump *Operator's Manual*.

For detailed instructions, warnings, specifications, and additional information for using the CADD®-Solis AC Adapter, refer to the *Instructions for Use* supplied with that product.

WARNINGS

To avoid serious personal injury, it is essential to observe the following warnings:

- **Do not expose the communication module to rain, water, or moisture. Shock or other hazard may result.**
- **Do not immerse the CADD®-Solis Communication Module rechargeable battery in liquid. Shock or other hazard may result.**
- **Do not use the CADD®-Solis Communication Module rechargeable battery with any device except the CADD®-Solis Communication Module.**
- **Do not connect any AC adapters directly to the CADD®-Solis Communication Module rechargeable battery. Directly connecting any AC adapter could be hazardous and can damage the rechargeable battery, communication module, and the pump.**
- **A CADD®-Solis Communication Module rechargeable battery that has reached the end of its useful life must be replaced with another CADD®-Solis Communication Module rechargeable battery. Using rechargeable batteries from other manufacturers could result in fire or explosion.**
- **Do not allow the electronic contacts of the communication module to simultaneously touch a piece of metal (a coin or paper clip, for instance).**
- **Do not use the CADD®-Solis Communication Module if it has been damaged, if internal electronics become exposed due to wear, or if the electronic contacts are damaged in any way. Fire, shock, or other hazard may result.**
- **Do not attempt to crush the CADD®-Solis Communication Module or the rechargeable battery. Fire, shock, or other hazard may result.**
- **Do not destroy the CADD®-Solis Communication Module rechargeable battery by incineration. It may explode.**
- **Do not use the communication module in the CADD®-Solis Infusion Pump pouch or any other pouch or bag. The communication module could overheat and power off utilizing the thermal protection circuit.**
- **There are potential health hazards associated with improper disposal of batteries and other electronics. Dispose of used batteries and other used accessories in an environmentally safe manner, and according to any regulations that may apply.**

CAUTIONS

- Do not use excessive force when separating the CADD®-Solis Communication Module cap from the base. The antenna cable must remain attached to both the cap and base.
- Disconnecting the antenna cable may result in damage to the antenna and/or communication module electronics.
- Failure to fully charge the CADD®-Solis Communication Module rechargeable battery prior to initial use and during prolonged storage may cause deep battery discharge, leading to premature battery replacement.
- The USB port on the CADD®-Solis Infusion Pump is intended for communication to the PharmGuard® Medication Safety Software, SureLink® Remote Support Software, and the CADD® Solis Network Setup Utility software only. Do not connect unsupported accessories to the USB port (e.g., charging other devices, attaching a wireless dongle) as this could damage the pump.
- If the CADD®-Solis Communication Module is stored outside of the environmental operating conditions and within the specified environmental storage conditions, allow the communication module to warm or cool to operating temperature for at least one hour prior to use to avoid damaging the electronic circuitry.

About The CADD®-Solis Communication Module, Model 2131


The CADD®-Solis Network Communication System (NCS) consists of a CADD®-Solis Ambulatory Infusion Pump, Model 2110 (software version 4.2 or greater), with a CADD®-Solis Communication Module installed and configured for your wireless network and PharmGuard® Server. When installed and appropriately configured, the communication module lets the pump wirelessly send pump history to a PharmGuard® Server, and to receive pump firmware and library updates.

The model 2131 is an upgraded communication module from the 2130. The radio is more robust, including better security and encryption and incorporating 20 MHz channel 'n' band support.

The CADD®-Solis Communication Module

includes the wireless radio and a rechargeable battery inside the module. The battery powers the communication module, and provides power for the CADD®-Solis pump when AC power is not readily available. The rechargeable battery is charged whenever the pump is plugged into AC power via the CADD®-Solis AC Adapter—even if the pump is turned off.

The **CADD®-Solis AC adapter** and **power cord** (available separately) is used to connect to the pump to recharge the battery when the communication module is installed in the CADD®-Solis pump.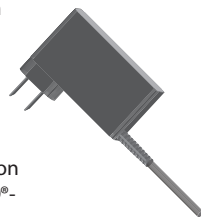

The **CADD®-Solis Network Setup Utility** software (available separately) is used to configure the wireless network settings for the specific pump when a CADD®-Solis Communication Module is installed. For information about using this software, refer to the installation instructions provided with the software.

NOTE: Version 1.1 of the Network Setup utility must be used.

WARNING: A CADD®-Solis Communication Module rechargeable battery that has reached the end of its useful life must be replaced with another CADD®-Solis rechargeable battery. Do not use replacement batteries from other manufacturers as that could result in fire or explosion. For more information, see the pump Operator's Manual, *Rechargeable battery reached end of use alarm*.

WARNING: Do not connect any AC adapters directly to the CADD®-Solis Communication Module rechargeable battery. Directly connecting any AC adapter could be hazardous and can damage the rechargeable battery, communication module, and the pump.


Installing the CADD®-Solis Communication Module

CAUTION: The installation and configuration of the CADD®-Solis Communication Module in a pump should only be performed by service personnel.


CAUTION: To avoid damage to the communication module, power off the pump and disconnect all cables connected to the pump prior to installing the communication module to the pump.

CAUTION: Because the initial installation of the communication module requires opening the module housing, this installation should only be performed at a workstation with electrostatic discharge (ESD) controls, including a grounded mat and wrist strap.


1. Make sure the CADD®-Solis pump is powered off and there are no cables attached to the pump.
2. Using your fingers, the pump key, or a coin, turn the knob on the battery door counter-clockwise and open the battery door.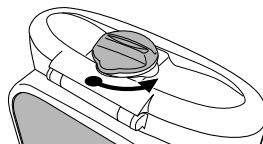


3. Remove any batteries that may be in the pump. Also, ensure that there is no debris in the battery compartment.
4. Pull the battery door rearward, as shown, to remove it from the pump. The battery door is not used with the CADD®-Solis Communication Module. Keep the battery door in an accessible location to allow the pump to be used later with AA batteries or the CADD®-Solis Rechargeable Battery Pack.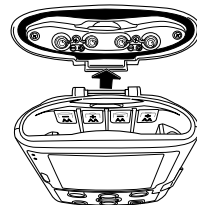


5. The three pogo pin insulators located on the three pogo pins in the pump battery compartment must be replaced with the insulators supplied with the communications module assembly. Three insulators are required and a fourth insulator is supplied as an extra. Discard the original insulators. Using a tweezers, carefully remove and discard the three original pogo pin insulators as shown.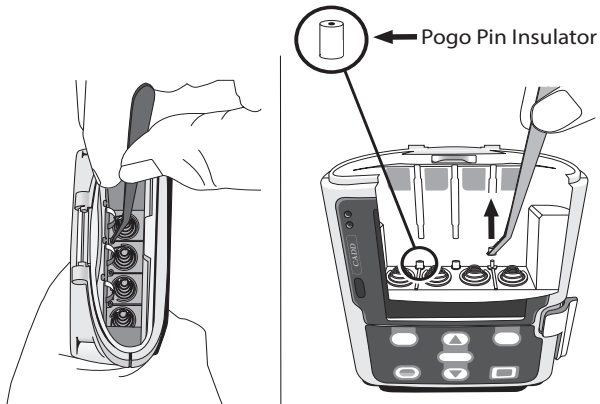

6. Install three new pogo pin insulators supplied with the communication module assembly.

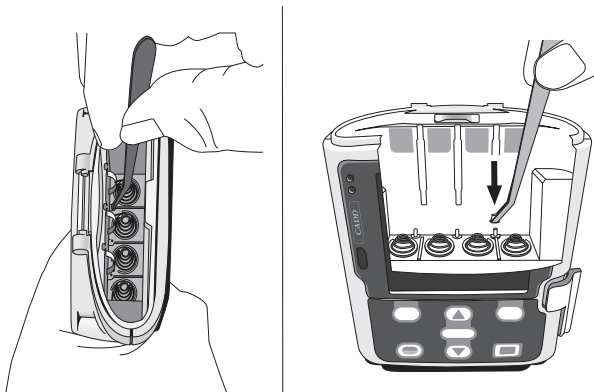
7. Verify the three new pogo pin insulators are fully seated around the pins.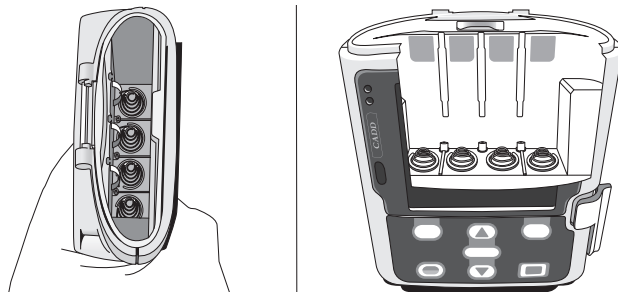

NOTE: The three new pogo pin insulators are also compatible with AA batteries and the CADD®-Solis Rechargeable Battery Pack. Discard the original pogo pin insulators as they are no longer needed.



8. **The rechargeable battery inside the communication module is not connected and must be internally connected.** To access the battery to connect it, use a 10 IP star-shaped drive bit to remove the two screws under the cap of the communication module.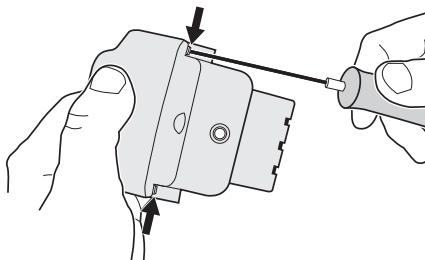

WARNING: To avoid potential injury and damage, do not remove the cap of the communication module while it is installed and connected to the pump or allow the electronic contacts of a Communication Module to simultaneously touch a piece of metal (a coin or paper clip, for instance).


9. Gently remove the cap from the base of the communication module, taking care to keep the antenna cable from pulling loose from either the cap or the base, and then **attach the battery plug to the battery receptacle on the communication module printed circuit board.**

CAUTION: Do not use excessive force when separating the communication module cap from the base. The antenna cable must remain attached to both the cap and base. Disconnecting the antenna cable may result in damage to the antenna and/or communication module electronics.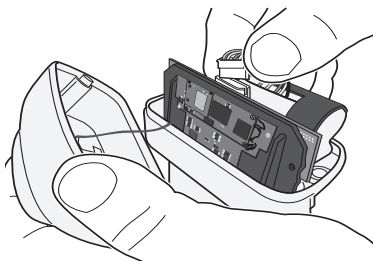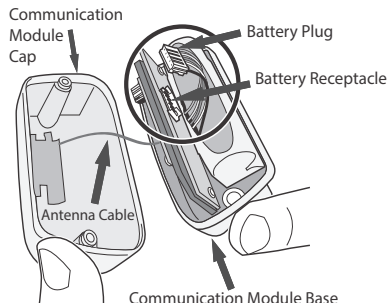


WARNING: Ensure that the seals around the module cap perimeter and screw holes are not lost or damaged as the module is opened or closed. These O-ring seals are necessary for water ingress and ESD protection.

10. Place the cap of the communication module onto the base, taking care to tuck the antenna cable completely inside the communication module.

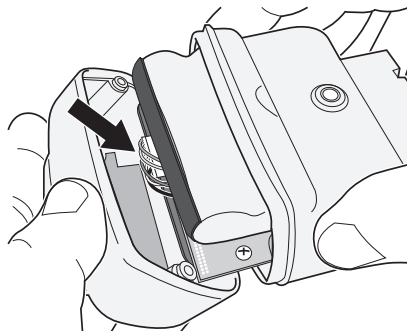
11. Insert and tighten the two star screws under the communication module cap using the 10 IP star-shaped drive bit until they are fully seated then another 1/8 turn (45 degrees) approximately. Do not over tighten.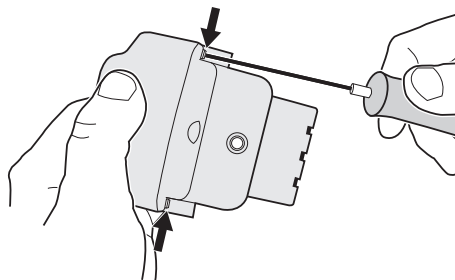


12. Insert the communication module into the pump battery compartment. Hook the bottom of the polemount adapter to the bottom of the pump, and then snap the top of the polemount adapter onto the base of the module.

NOTE: Align the teeth of the polemount adapter to the recessed opening in the pump housing.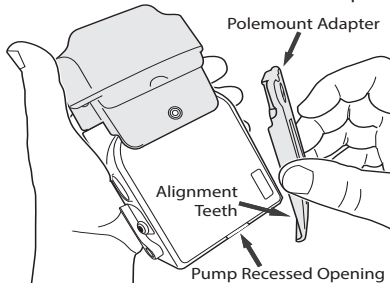

13. Use a CADD®-Solis Pump Key, standard screwdriver, or a coin to tighten the screw to attach the polemount adapter and secure the communication module to the pump. Tighten the screw until it is fully seated then another 1/8 turn (45 degrees) approximately. Do not over tighten.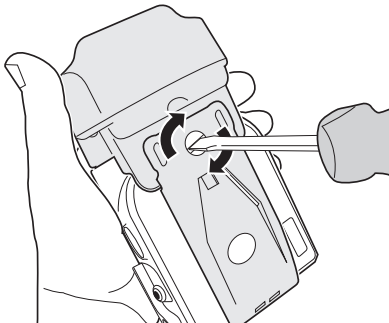

14. **Before configuring the pump and first use, the communication module will need to be fully charged.** See the next section for more information about charging the rechargeable battery in the CADD®-Solis Communication Module.

15. If the CADD®-Solis Communication Module is being installed into a specific pump for the first time, the wireless settings for the pump will need to be configured using the CADD®-Solis Network Setup Utility software (available separately). For more information, see the documentation provided with the network setup utility software.

Charging the Rechargeable Battery

The communication module rechargeable battery is charged whenever the CADD®-Solis pump is plugged into AC power via the CADD®-Solis AC Adapter—even when the pump is turned off.

Within the operating temperature range of 2°C to 40°C (35°F to 104°F), the rechargeable battery will become fully charged in 10 hours or less.

Fully charge the rechargeable battery:


- Before the **first use**
- At least **every 30 days** while in regular use
- At least **once every 6 months** while in storage
- When the pump indicates the battery is low or depleted

CAUTION: Failure to charge the rechargeable battery during prolonged storage may cause deep battery discharge, leading to premature battery replacement.

CAUTION: To avoid damaging the AC adapter connector or the pump, do not use excessive force or instruments such as pliers to attach or detach the AC adapter connector.

To charge the rechargeable battery:

1. Plug the power cord for the CADD®-Solis AC adapter into an AC outlet and the other end into the AC adapter.
2. Plug the AC adapter connector into the AC power jack (labeled “7V”) on the CADD®-Solis pump. When properly connected, the blue light next to the connector on the pump lights up.

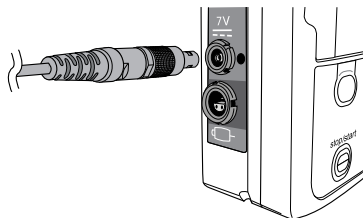WARNING: Do not connect any AC adapters directly to the CADD®-Solis Communication Module rechargeable battery. Directly connecting any AC adapter could be hazardous and can damage the battery, communication module, and the pump.

CAUTION: Failure to push the AC adapter output connector all the way inward to the pump may result in an intermittent connection and the connector may dislodge, causing a loss of power and will no longer charge the rechargeable battery.

3. When the pump is powered on, the pump status bar briefly shows “AC Adapter Connected.”

AC Adapter Connected

4. When the pump is powered on, the pump

status bar shows that the pump is connected to AC power. A small plug indicator is shown to the right of the battery icon.

5. When the pump is powered on, you can assess the estimated battery power level from the status bar on the pump display. For more information, refer to the pump *Operator's Manual*.

When the battery is fully charged, you may choose to disconnect the AC adapter plug. The “AC adapter disconnected” message is briefly displayed after disconnecting the plug.

Configuring Wireless Settings Using the CADD®-Solis Network Setup Utility Software


All wireless settings for the pump and CADD®-Solis Communication Module are configured using the **CADD®-Solis Network Setup Utility** software (available separately) via a USB connection from a PC to the CADD®-Solis pump.

The CADD®-Solis Network Setup Utility software lets you select and enter all wireless settings for configuration to your wireless network access points, and your PharmGuard® Server. Those settings can be saved to the CADD®-Solis pump. Therefore, if a CADD®-Solis Communication Module is replaced for the pump, the new communication module will automatically use the pump's pre-configured wireless settings.

For more information, see the documentation provided with the CADD®-Solis Network Setup Utility software.

Turning Wireless Radio On/Off

When the CADD®-Solis Communication Module is installed, its wireless radio can be turned on or off as necessary (e.g., turn off to conserve battery, etc.). This can be done at any time from the **Tasks** menu on the pump screen without interrupting clinical functions of the pump such as a running infusion. For more information, see the pump *Operator's Manual*.

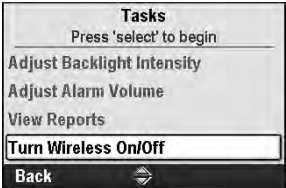Additionally, you can view detailed wireless status and wireless settings. From the pump **Reports** menu, select **Wireless Status** or **Wireless Settings**. To view communication module (CM) version and MAC address, select **Device Information** from the Reports menu. For more information, see the pump *Operator's Manual*

Viewing Wireless Status and Settings

While the CADD®-Solis Network Setup Utility software lets you view and change wireless settings, you can also view wireless status and settings on the CADD®-Solis pump screen. Wireless status is indicated on the pump main screen:

	Wireless Off (white indicator)
	Wireless On, not associated with wireless access point (orange indicator)
	Wireless On, associated with wireless access point (orange indicator with green center)
	Wireless On, associated with wireless access point, and communicating with PharmGuard® Server (green indicator with green center)

Troubleshooting

Problem	Probable Cause / Solution
The rechargeable battery won't charge.	- The AC adapter connector may not be completely inserted into the connection port on the pump. Fully insert the AC adapter connector into the pump. - The battery may be too hot or too cold. Make sure the environmental conditions meet those specified for operation. - The three replacement pogo pin insulators are not installed or are improperly installed.
The rechargeable battery is fully charged but won't work in the pump.	There may be a problem with contact between the pump and communication module. - Make sure the contact pads on the communication module are clean. - Make sure the battery compartment and the contacts on the pump are clean. - Try the communication module in another pump. If it works in the second pump then there is likely a problem with the first pump. If the communication module still doesn't work in the second pump, there is likely a problem with the communication module and/or the rechargeable battery. Contact Smiths Medical Customer Service.
Wireless radio is turned on and the pump wireless status icon is displaying "Wireless On, not associated with wireless access point" (orange indicator).	- The Communication Module may be in heat reduction mode, see the pump <i>Operator's Manual</i>. - Wireless settings are improperly configured. Check and/or configure settings. - Communication module is too far from a wireless access point. Move closer to an access point. - Pump is automatically turning off the radio due to a depleted battery. Connect pump to AC Adapter. - Communication module in vicinity of an operating microwave oven. Move away from microwave oven.

Pump Alarms Associated with the Communication Module

Rechargeable battery near end of use. Replace battery.	The rechargeable battery pack is near the end of its life. It has been discharged and recharged so many times that it will soon be at the end of its use. You may continue to use it in this state.
Rechargeable battery reached end of use. Pump will not run.	The CADD®-Solis Communication Module rechargeable battery pack is at the end of its life. It has been discharged and recharged so many times that it is no longer able to hold a good charge. Replace with a new CADD®-Solis Communication Module rechargeable battery.
Battery Unusable. Pump will not run.	The battery is not compatible with the pump or a communication error occurred. For more information, see the pump <i>Operator's Manual</i> .
Communication module intermittent connection to pump	The connection between the communication module and the pump is intermittent. For more information, see the pump <i>Operator's Manual</i> .

Transportation

The communication module has been tested to and complies with the United Nations transportation regulations of UN/DOT ST/SG/AC.10/11 (Rev. 5) Section 38.3 testing requirements T1-T5, T7.

Disposal / Collect Separately

The communication module rechargeable battery should be completely discharged prior to disposal and/or the terminals should be taped or capped to prevent short circuit.

Non-European Union residents must dispose of or recycle this product in accordance with the local laws or regulations that apply.

Collect Separately

This product contains electrical and electronic components that may contain materials, which if disposed of with general waste, could be damaging to the environment.

In accordance with Directive 2002/96/EC Waste Electrical and Electronic Equipment, residents of the European Union must follow specific disposal or recycling instructions for this product. Contact your local distributor, or visit the following web site for specific instructions:

<http://www.smiths-medical.com/recycle/index.html>

WARNING: There are potential health hazards associated with improper disposal of batteries and other electronics. Dispose of used batteries and other used accessories in an environmentally safe manner, and according to any regulations that may apply.

REACH Information

Under the European Regulation for Registration, Evaluation, Authorization and Restriction of Chemicals ("REACH Regulation EC No. 1907/2006") companies are required to disclose to customers if a product contains any of the substances identified on the Candidate List of Substances for Authorization (also known as the Substances of Very High Concern (SVHC)) at a concentration above 0.10% by weight. Smiths Medical's declarations regarding SVHCs in products can be found at: www.smiths-medical.com/environment

Part 15 Certification and RSS standard(s)

This device complies with part 15 of the FCC Rules and Industry Canada license-exempt RSS standard(s).

Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Cleaning and Disinfecting the Communication Module

For optimal efficiency, it is recommended best practice to clean first and then disinfect. All disinfectants require pre-cleaning, except for disinfectant cleaners that are tested to disinfect in the presence of 5% of organic matter.

CAUTION: Do not immerse the communication module in cleaning fluid or water. Do not allow solution to soak into the communication module. Moisture buildup inside the communication module may damage the device. Clean and disinfect the exposed surfaces of the communication module when installed in the pump.

CAUTION: To avoid damage to the communication module internal electronics, do not allow solution to soak into the electronic contacts area or in any openings and seams. Use only a dry lint-free cloth to clean the electronic contacts and the area surrounding normally enclosed when installed in the pump.

The following steps may be used to clean and disinfect the communication module:

1. Clean the communication module using a mild detergent soap solution to remove residuals or contaminated material. Apply solution to a soft, lint-free cloth and then wipe the exposed surfaces of the communication module when installed in the pump. *Do not allow the solution to soak into the pump or the communication module.*
2. Disinfect the communication module by applying a disinfecting solution (listed below) according to the disinfecting product label instructions. If using a liquid or spray, apply solution to a soft, lint-free cloth and then wipe the exposed surfaces of the communication module when installed in the pump. Follow the disinfectant manufacturer's recommendations for disinfectant contact times. *Do not allow the solution to soak into the pump or the communication module.*

Acceptable disinfecting solutions for the CADD®-Solis Communication Module are listed below.

Product	Manufacturer (EPA Registration Number)	Active Ingredient(s)	Contact/ Kill Time
CaviWipes™	Metrex (46781-8)	17.2% Isopropanol	3 minutes
Sani-Cloth® Super	PDI (9480-4)	Dimethyl Benzyl Ammonium Chloride, Dimethyl Ethyl Benzyl Ammonium Chloride	2 minutes
Sani-Cloth® Bleach	PDI (9480-8)	0.60% Sodium Hypochlorite	4 minutes

3. Allow the communication module to dry completely before use.

Specifications

Battery type

- Rechargeable Lithium-ion battery (2 cells)
- Equivalent Lithium content < 1.6 g

Battery capacity

- 18.7 Wh (when fully charged)

Battery output

- 3.6V (nominal)

Rechargeable battery weight

- < 100 g

Operating environmental requirements

- Temperature 2°C to 40°C
(35°F to 104°F)
- Humidity..... 20% to 90% relative
humidity,
non-condensing
- Atmospheric pressure 70 kPa to 106 kPa
(10.2 psia to 15.4
psia)

Storage and transportation environmental requirements

- Temperature -20°C to 60°C
(-4°F to 140°F)
- Humidity..... 20% to 90% relative
humidity,
non-condensing
- Atmospheric pressure 70 kPa to 106 kPa
(10.2 psia to 15.4
psia)

Product battery life

- 400 full cycles (charge / discharge) at room
temperature, 23°C (73°F)

Maximum charge time (fully depleted battery to fully charged)

- 10 hours when plugged into AC Adapter

Pump and communication module run time (new battery)

- 4 hours at pump nominal settings (see the
pump *Operator's Manual* for nominal settings
information)

Safety

- The communication module complies with Underwriters Laboratories UL 2054, IEC 60601-1(2005 + A1:2012) Ed 3.1 [when installed in the CADD®-Solis Model 2110 infusion pump], IEC 60601-1-2 (2014), and IEC 62133 (2002).
- The lithium-ion cells comply with Underwriters Laboratories UL 1642.

IEEE 802.11 a,b,g,n Radio for Network Communication

- The CADD®-Solis Communication Module intentionally receives and transmits Radio Frequency (RF) electromagnetic energy for non-critical communication.
- RF transmit / receive, "a" band 5180 MHz to 5320 MHz & 5500 MHz to 5825 MHz
- RF transmit / receive, "b", "g" and "n" bands...2412 MHz to 2472 MHz
- Bandwidth of "a", "b", "g" and "n" signals20 MHz
- Maximum RF transmit power of band "a"10mW
- Maximum RF transmit power of bands "b", "g" and "n"100mW

Note: The communication module may be interfered with by other equipment, even if that other equipment complies with CISPR 11 requirements.

For additional CADD®-Solis Communication Module network specifications and setup information, see the Network Setup Utility Installation and User Guide (Version 1.1).

Interference / Separation Distances

Other electronic communication devices used in the vicinity of the communication module may cause interrupted transmission of data. Interrupted transmission of data does not impact pump delivery. The table below specifies what effect each type of device causes and describes the minimum separation distance that should be observed.

Interferer (type of electronic device)	Min. Separation Distance
2.4GHz/5GHz Wireless Transmitter Base Stations (e.g. Wi-Fi® Access Points, Wireless Telephone Base Stations)	3 meters (9.84 feet)
2.4GHz/5GHz Wireless Medical Devices and other Peripherals (e.g., Wi-Fi® enabled patient monitors, infusion pumps, computers/smart-phones/tablets; Bluetooth-enabled devices; Zigbee-enabled devices; Wireless telephone handsets)	1 meter (3.28 feet)
Other Smiths Medical CADD®-Solis pumps with Wireless Connectivity	0.3 meters (1 foot)
433MHz/2.4GHz/5GHz RFID Asset Location Tags	Direct contact

Communication Module Wireless Range:

36.58 meters (120 feet) (line-of-sight operation with access point)

Network Modes and Encryption

Network Modes:	Encryption Modes:
• 802.11 a/b/g/n	• None
• 802.11 b/g/n	• TKIP
• 802.11 a/n	• CCMP
	• TKIP/CCMP

Authentication Modes:	EAP Modes:
• OPEN	• None
• WPA-ENT	• EAP-TLS (v1.0 and v1.2)
• WPA2-ENT	• EAP-TTLS
• WPA-PSK	• EAP-PEAPV0 (PEAP with MS-CHAPv2)
• WPA2-PSK	• EAP-PEAPV1 (PEAP with GTC)
	• EAP-LEAP
	• EAP-FAST

Certificates:

SSL SHA-1
SSL SHA-2

Limited Warranty

Warranty: Smiths Medical ASD, Inc. ("Manufacturer") warrants to the Original Purchaser that all parts and components of the CADD®-Solis Communication Module(the "Communication Module") that are manufactured by or for the Manufacturer, excepting those items indicated below, shall be free from defects in materials and workmanship under normal use, if used in accordance with the Instructions for Use, for a period of one (1) year from the actual date of sale to the Original Purchaser. THERE ARE NO OTHER WARRANTIES.

Exceptions: This warranty does not cover normal wear and tear and maintenance items, and specifically excludes any other accessory items or equipment used with the Communication Module.

Warranty Procedures: Subject to the conditions of and upon compliance with this Limited Warranty, the Manufacturer will repair or replace at its option without charge (except for a minimal charge for postage and handling) any Communication Module (not including accessories) that is defective if a claim is made during such one-year period. DO NOT send any Communication Module or component in to the Manufacturer for warranty repair without specific forwarding information from the Customer Service Department.

Conditions: The following conditions, limitations and exclusions apply to the Manufacturer's obligations under this warranty:

A. Voiding of Warranty: This warranty is null and void if the Communication Module, or any part or component thereof, has been (1) repaired by someone other than the Manufacturer or its authorized agent; (2) altered so that its stability or reliability is affected; (3) misused; or, (4) damaged by negligence or accident. Misuse includes, but is not limited to, use not in compliance with the Instructions for Use of the Communication Module or use with non-approved accessories.

B. Limitations and Exclusions: Replacement of the Communication Module or any component part is the EXCLUSIVE remedy offered by the Manufacturer. All recommendations, information and descriptive literature supplied by the Manufacturer with respect to the Communication Module are believed to be accurate and reliable, but do not constitute warranties. No agent, representative, or employee of the Manufacturer has authority to bind the Manufacturer to any representation or warranty, expressed or implied. THERE IS NO WARRANTY OF MERCHANTABILITY OR FITNESS OF THE COMMUNICATION MODULE FOR ANY PARTICULAR PURPOSE.

THE MANUFACTURER DISCLAIMS RESPONSIBILITY FOR THE SUITABILITY OF THE COMMUNICATION MODULE FOR A PARTICULAR MEDICAL TREATMENT OR FOR ANY MEDICAL COMPLICATIONS RESULTING FROM THE USE OF THE COMMUNICATION MODULE. THE MANUFACTURER SHALL NOT BE RESPONSIBLE FOR ANY INCIDENTAL DAMAGES OR CONSEQUENTIAL DAMAGES TO PROPERTY, LOSS OF PROFITS, OR LOSS OF USE CAUSED BY ANY DEFECT OR MALFUNCTION OF THE COMMUNICATION MODULE.

This warranty gives the Original Purchaser specific legal rights, and the Original Purchaser may have other legal rights that may vary from state to state.

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