

# **EveLab Insight - Eve M Skin Imaging Analysis System**

## **User Guide**

### **Contents**

Product Introduction  
Packing List  
Parts List  
Device Status  
Manual  
Safety Precautions  
Warranty/Repair  
Warranty Card  
Specifications

### **Product Introduction**

MeituEve M is a professional skin analysis system. With built-in 5 spectra and 3 cameras, it can complete a skin analysis of more than 20 dimensions in 12 seconds, providing customers with an accurate and comfortable skin analysis experience.

### **Packing List**

Main device: 1 Unit  
Lens hood: 1 Piece  
Tablet base: 1 Piece  
Power adapter: 1 Piece  
User guide: 1 Copy  
Please use the original accessories.  
Official website: [www.evelabinsight.com](http://www.evelabinsight.com)

### **Parts List**

Ambient light  
Chin support  
Forehead support  
Type-C connector  
Power connector  
Standby Light

## Device Status

Power on: The ambient lights on both sides are on for 3 seconds.

The network is waiting to be connected: The ambient lights on both sides are flashing.

The network is connected: The ambient lights on both sides are always on.


Power off: The ambient lights on both sides are off.

## Manual

### I. Unboxing Guide

1. Take out the main device: Place the main device on the desktop according to the instructions on the box.(Please do not grasp forehead support or chin support and do not touch the color card on the back of the chin support).

2. Install lens hood: The hood contains magnets. Please attach the hood to the front of the main device according to the instructions below. The logo on the front of the hood should be parallel to the desktop.

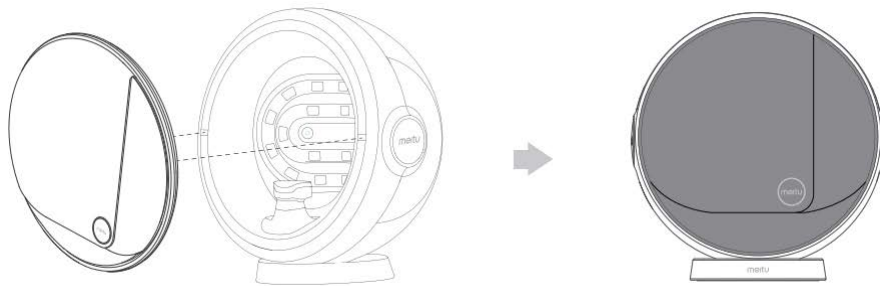

3. Connect to the power supply: Connect the power cord to the power jack on the back of the main device."

### II Power and Connection

1. Power on: Short press the power button on the back of the main device. The ambient lights on both sides of the main device are on for 3 seconds, and there is a sound indicating that the booting has been completed.

2. Download: Open the App Store on your tablet and search for the app “EveLab Insight - Eve M”. Download and install it.

3. Log in: Open the app and log in with the store account and password.

4. Connect: Place the tablet close to the device and connect according to the prompts from the software interface.

### III Take Photos and Analyze Skin

1. Create profile: Create a profile for customers according to the prompts in the software interface.

2. Clean: Wipe the front of the forehead support and chin support with a cotton alcohol pad. (Do not let alcohol or other liquids touch the color card on the back of the chin support)

3. Prepare to take photos: After the customer cleans their face, make sure their face is not blocked by bangs, jewelry, glasses, or any other items. Then, have the customer sit in front of the

device with their chin resting on the chin support and forehead resting on the forehead support, making sure their head is straight and eyes are closed. The operator should then cover the customer's head with the lens hood and ensure only the customer's face is displayed on the screen before taking photos.

4. Take photos and analyze skin: The operator conducts the skin detection according to the prompts in the software interface.
5. Explain the report: After the skin detection is completed, the operator can show and explain the skin analysis report to the customer via your tablet.
6. Power off: Short press the power button.

#### IV. Maintenance Instructions

1. Before each detection, the operator needs to use a cotton alcohol pad to wipe the front of the forehead support and chin support to ensure the cleanliness of the device for each customer.
2. The casing for the device is made from environmentally-friendly materials. If it needs cleaned, please wipe it with an alcohol pad.

※Caution: Do not touch the lens or the color card on the back of the chin support with fingers or alcohol pads.

3. Please keep the detection camera and luminaire clean. Otherwise, the performance of the product may be adversely affected. "

## Safety Precautions

The precautions herein are to ensure proper use of the device and prevent personal injury and property damage to users and others. Please strictly follow the precautions.

1. Only use the original Eve M power adapter with this product. Otherwise, power related problems or damage to the device may occur;
2. During use, keep the power plug securely attached to the power outlet and make sure it is easily accessible to plug and unplug;
3. If the power adapter is used incorrectly, the power cord may break. Do not use the device if the power cord is twisted and cannot be restored, or if the power cord or the power connector is damaged, deformed or broken;
4. When the device is not in use for long period of time, store the device in a dry, dark, cool place and avoid contact with hard objects so as not to damage the product;
5. Do not store the product and its accessories in a humid environment for a long period of time. Otherwise, the performance of the product may be adversely affected;
6. Please keep this device away from any heat sources or open flames;
7. Do not immerse the device in water or rinse with water;
8. Please store the device in a place that is out of reach of children;
9. Do not disassemble, repair, or rebuild this device without authorization;
10. Do not unplug the power cord while the power is on.
11. This product is a precision instrument, please do not rough handling, vibration, knocking this product, so as not to cause damage."

This device contains strong magnets. The following individuals (including but not limited to)

should not use this product:

1. recipients of pacemaker surgery;
2. those that are suffering or have suffered from retinal detachment;
3. those with facial implants, metal or otherwise, due to fractures, plastic surgery, etc.;
4. those with residual effects of head or eye injuries due to traffic accidents, etc.;
5. those with doctor's orders to refrain from contact with strong magnetic force.

## **Warranty/Repair**

I. Starting from the date of purchase, a one-year warranty service is provided upon presentation of invoice, shopping list, and warranty card. During the warranty period, if any issues with the device arise as a result of product quality, you will be entitled to free repair and maintenance services.

II. The following conditions are not covered in the scope of warranty:

1. Products not purchased from official channel;
2. Warranty period exceeded;
3. Damage caused by unauthorized dismantling or repair;
4. Burning of the circuit board due to human fault, such as immersion in liquid or dropping;
5. Damage caused by improper installation, use, maintenance or storage;
6. Warranty certificate not in conformity with product model or warranty certificate being altered;
7. Damage caused by force majeure.

## **Warranty Card**

S/N

Customer

Name

Phone Number

Address

Invoice Number

Purchase Date

Dealership/Entity

Name

Address

Phone Number

Specifications

## **FCC Statement**

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and  
(2) This device must accept any interference received, including interference that may cause undesired operation.

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

RF warning:

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with minimum distance 20cm between the radiator & your body.

Brand name: EveLab Insight

Product name: MeituEve M

Product model: MS1901

Color: Aurora white

Material: PC / silicone / metal

Operating temperature: 0°C - 40°C

Overall machine size: 580mm\*420.5mm\*601mm

Rated voltage: 12V

Execution standards: GB4706.1-2005, GB4706.15-2008