Wireless Visual Ear
Pick manual

Please read this manual
carefully before use
and keep it properly for
future reference.

| APP download

Please use your mobile browser to
scan the QR code for downloading
and installation.

PicidaePro

► iOS app store

Android Google

Android China

Apple users
Select the los appstore option
after scanning the code.Open
the App Store to download and
install.
You can also directly search for
PicidaePro downloads in theApp
Store.

PicidaePro

IOS app store

► Android Google

Android China

Android users
A fter scanning the code, select
the Android Google option to enter
the next page,Click: Download
and install through third-party
browser options.

| APP Connection:

1. [Power on]
Press and hold the oval power button 2
at the bottom of the product body
seconds to boot;
2. [Open APP]
Click the Start button on the left and
wait for it to connect automatically
Connected products;
3. [Get Started]
Once the connection is successful, click
on the left side again to start
button to visualize the ear canal.

| Troubleshooting:
Screen freezing:

* Make sure the product is used in
close proximity to the phone within the
Wi-Fi signal transmis-sion range.

* Low battery can weaken the Wi-Fi
signal, so please charge it in a timely
manner.

*If there are many Wi-Fi signalsin the
environment, they may interfere with
the signal of the product. Please use it
in an area with fewer Wi-Fi signals.

No image display:

* Confirm that the app has been granted
permission to use network data.

* Check if the phone is connected to
the product's Wi-Fi network. If
connected but still no display, try
closing and reopening the app.

Unable to store images
or videos:

* Confirm that the app has been
granted permission to take photos and
record videos.

Unable to charge:

* Please use the provided charging
cable to charge.

* Try using a different charger if the
problem persists.

| Charging Instructions:

1. Before initial use, please ensure that
the product is fully charged.
- 2.It is not recommended to completely
drain the battery.
- 3.When the indicator light flashesred,
please charge it promptly
- 4.When charging, connect the Type-C
interface plug to this product.

| Indicator Light
Instructions:

1. After powering on, the blue light will
flash.
- 2.When connected to Wi-Fi, theblue
light will stay on.
3. When the battery is low, the
redlight will flash.
4. During charging, the red light
willstay on.
5. When the battery is fully
charged,the red light will turn off.

| Precautions

- 1.Please turn off the product before
charging. Do not use a fast charger
for charging.
2. If the image is unclear, please use
a cotton swab to wipe the lens.
- 3.Before use, observe the
surrounding environment to prevent
accidents.
4. It is normal for the lens and body to
generate heat.
- 5.This product is for personal care
use only and should not be used as a
medical device.
- 6.Please keep the product in a cool,
dry place, away from fire and
corrosive substances.
7. This product is not suitable for
children.
8. Do not immerse the product in
water or rinse it under a faucet.
9. Do not allow children to play with
this product.
10. Do not take the product to an
unauthorized service center for
repairs.
11. Do not use any accessories or
partsproduced by other
manufacturers, using such
accessories or parts will void your
warranty.

FCC Compliance Statement: This
device complies with Part 15 of the
FCC Rules. Operation is subject to
the following twoconditions:(1)this
device may not cause harmful
interference, and (2)this device must
accept any interference
received,including interference that
may cause undesired
operation.Changes or modifications
not expressly approved by the party
responsible for compliance couldvoid
the user's authority to operate the
equipment.

NOTE: This equipment has been tested and
found to comply with the limits for a Class B
digital device, pursuant to Part 15 of the FCC
Rules.These limits are designed to provide
reasonable protection against harmful
interference in a residential instak-ation. This
equipment generates,uses instructions, may
cause harmful interfe-rence to radio
communications.However there is no
guarantee that interference will not occur in a
particular installation. If this equipment does
cause harmful interference to radio or
television reception, which can be determined
by turning the equip-ment off and on,the user
is encouraged to try to correct the interference
by one or more of the following measures:
--Reorient or relocate the receiving antenna.
--Increase the separation between the
equipment and receiver.
--Connect the equipment into an outlet on a
circuit different from that to which the receiver
is connected.
--Consult the dealer or an experienced
radio/TVtechnician for help.

| Card After-sales Warranty

Product Warranty Policy:
1.Under normal use, the warranty period for
this product is one year.
2. The following situations are not covered by
the warranty (including but not limited to): a. No
warranty card or any proof of purchase. b.
Exceeding the scope of precautions.abnormal
use. c.Self-repair or damage caused by human
factors(including improper operation), aswell as
damage caused by natura disasters. d.
Exceeding the "Three Guarantees" period, or
using the product for purposes beyond its
intended use.
3. When returning the product for repair, please
include the warranty card.
4. The company reserves the right to modify the
warranty terms and the final interpretation of
the terms.

After-sales Warranty
Card

Customer Name _____

Customer Phone _____

Customer Address _____

Product Name _____

Product Specifications/Model _____

Remarks _____

Thank you for choosing our product. All our products
undergo rigorous quality testing before leaving the
factory. If you have any issues with the product,please
contact our after-sales service. We are committed to
providing you with assistance.

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