Bedienungsanleitung
Operating Instructions
Mode d'emploi
Istruzioni d'uso
Manual de instrucciones


DE

EN

FR

IT

ES

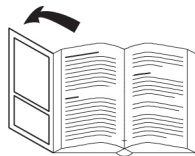Deutsch

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English

About these instructions.....	32
Safety	32
Technical specifications	34
Product description	36
Operating and functional elements	38
Download the app	39
Commissioning	40
optaro app	43
Troubleshooting	50
Care instructions	53
Disposal	53
Applicable standards.....	54
Warranty.....	58
Trademarks	58

About these instructions

This quick reference guide contains the most important information about your product. You can access the complete instructions on our homepage <https://www.eschenbach-vision.com/en-GB/products/product/optaror-16533> or in the app in the "Operating instructions" section.

Read these instructions carefully and thoroughly before using this product.

Keep these instructions for future reference.

These instructions are based on the app version "1.0.0".

Safety

Intended use

This electronic reader is intended exclusively for enlarging and reading or viewing texts or images. To ensure proper function, the reader must be connected to a compatible iPhone via a separately available smartphone protective cover or a universal stand.

This product is intended for use in a domestic environment (including care homes, workplaces, schools and other educational establishments).

Important information about the usage environment

The device connects to the iPhone via WiFi. The use of WiFi and the corresponding radio connections are defined by national regulations.

This device is approved for use in the following countries:
DE, AT, BE, BG, CY, CZ, DK, EE, EL, ES, FI, FR, HR, HU,
IE, IS, IT, LI, LT, LU, LV, MT, NL, NO, PL, PT, RO, SE, SI,
SK, CH, USA, UK(NI)

In Japan, use is only permitted indoors. Please observe the respective national regulations.

No liability is accepted for improper or unauthorised use in non-approved countries.

Scope of delivery

- optaro
- USB cable (Type-C to Type-C)
- Operating instructions

Technical specifications

Magnification range	3.3x or 3.7x to 15x (depending on the end device used)
Display mode	True colour 14 contrast-enhanced contrast modes
Camera	Fixed focus at a distance of 24mm
Resolution	1920 x 1080
Illumination	LED
Recommended power source	USB-PD (5V===1.8 A/9 V===1.0 A/12 V===0.75 A)
USB connection	Type-C
WiFi SSID	See type plate and operating instructions
Conducted power	<9dBm

Radio network	WLAN 5745 - 5825 MHz (EU, UK, US), 5180-5240 MHz (Japan)
Operating time	Approx. 2 hours
Charging time	Approx. 1.5 hours
Dimensions (folded)	103 x 72 x 20 mm
Weight	120 g
Operating temperature range	0°C - 35°C

Product description

The optaro electronic reader is a medical device for people with impaired vision and is used to magnify reading material (e.g. text and images).

optaro (without smartphone protective cover)

optaro (with smartphone protective cover)

The product is connected to a compatible iPhone using a separately available protective smartphone case.

The connection to the smartphone is established via a local WiFi connection. No internet connection is required for this.

The reader is powered by its own battery. The battery is charged via USB-C and has an operating time of approx. 2 hours.

The foldable stand holds the reader in position

The product is controlled via the “optaro app”.

The reading material is recorded via the reader’s camera. The app magnifies the recorded data on the smartphone display. The reader enables 3x to 15x magnification (depending on the mobile device), various false colour modes and is equipped with a fixed-focus camera.

Other functions include the photo function, a read-aloud function of individual words or groups of words, the display of a reading aid and an overview display.

Compatible smartphones

optaro protective cases are available for a variety of different iPhone models. A list of all compatible devices can be found under the following link:

<https://www.eschenbach-vision.com/en-GB/products/product/optaror-16533>

Operating and functional elements

(For figures, see fold-out pages A1 and A2 left and right)

- A Swivelling camera
- B RESET button
- C Status LED
- D On / Off button
- E LED illumination
- F Retractable stand
- G Eyelet for neck strap
- H Charging indicator LED
- I USB-C charging port

Download the app

The optaro app is freely available in the Apple App Store and can be downloaded there.

Alternatively, the optaro app can be downloaded using the QR code on the back.

EN

Important information about the QR code

- Keep the QR code in a safe place for future use (e.g. changing the device)!

Commissioning

Information on the smartphone cases

The reader must be used with the appropriate protective cover or the universal stand.

To set up the optaro reader, proceed as follows:¹

1. Download the optaro app from the Apple App Store or via the QR code on the back.
2. Switch on the reader by pressing the on/off button (D).
 - The status LED above lights up blue.
3. Start the optaro app on your smartphone.
4. Press “Next” in the optaro app.
5. Confirm that you have read and agree to the terms of use by ticking the box. You must agree to the terms of use in order to use the app and the reader.

¹ For information on installing the optaro and protective cover, please refer to the installation instructions for the corresponding protective cover.

6. Press "Next".

Note

To be able to use the optaro with the app, a local WiFi connection is established between the iPhone and optaro. iOS requires permission to find WiFi devices in the vicinity.

EN

For the initial setup, you will be asked to enter the optaro SSID once. This can be done in two ways:

- Manual entry of the SSID (1)
 - Scanning the QR code on the type plate (2)
 - ① If you want to establish the connection by scanning the QR code, you must authorise the optaro app to access your iPhone's camera.
7. Enter the SSID in the field provided above or use the scan function (2).
8. Press "Next".
9. The connection must be authorised during the setup process.
- The connection is established with the camera module.
 - After a successful connection, the status LED (C) of the camera module changes from blue to green.

- The main screen of the app is displayed.
- Now place the smartphone in the protective cover.
- ✓ The reader is ready for use.

Establishing the connection between the smartphone and optaro


For regular use, the connection between the iPhone's WiFi and the optaro camera module must be authorised each time.

optaro app

Display screen

The main screen of the optaro app is structured as follows and offers various functions:

EN

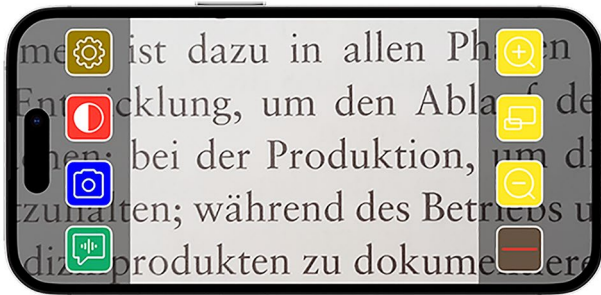Icon	Name	Meaning
	Zoom in	Enlarges the font
	Overview function	Switches between the smallest and the previously set magnification
	Zoom out	Reduces the font
	Activate/deactivate reading support	Activates/deactivates reading support
	"Tell me" function	Starts the read aloud function for the central reading area

Icon	Name	Meaning
	Take a photo	Take a photo and switch to photo mode
	Activate/deactivate false colours	Switch between the selected false colour
	Settings	Opens the “Settings” menu

Functions

Settings

You can make various settings. For example, you can change the operating mode, adjust reading support, change the lighting or select additional false colours.

You can find a detailed explanation in the “Operating instructions” chapter in the app or in the downloadable instructions.

False colours


To display the camera image in false colours, switch between the various false-colour modes or return to the true colour display, press repeatedly .

Take a photo

In photo mode, you can save the current screen section to your gallery, open the gallery or share the displayed screen section.

optaro saves the photos in a separate album named “optaro” (iOS requires permission to access the photo library).

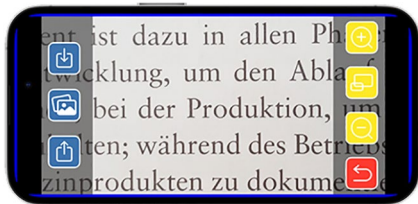EN

Save

Saves the displayed screen section as a photo in the gallery.

Photo gallery

Opens the “Photo gallery” of the iPhone

Share photos

Opens the menu for sharing and sending the photo


Back button

Exits the photo mode and switches back to the live image

“Tell me” function

You can use the “Tell me” function to have a central group of words read aloud. The central word group is read aloud by pressing the  button. The area read aloud is highlighted with a green border.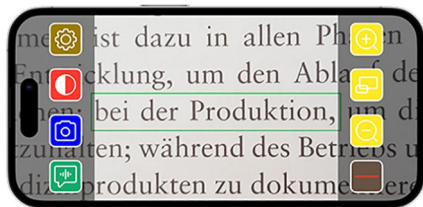

Important information on voice output

The function is not available in all languages.¹

¹ Available languages are: German, English, French, Italian, Spanish and Portuguese


Zoom in /

Zoom out

You can use the “Zoom in / Zoom out” function to enlarge or reduce the current screen section or switch between the smallest and previously set magnification.

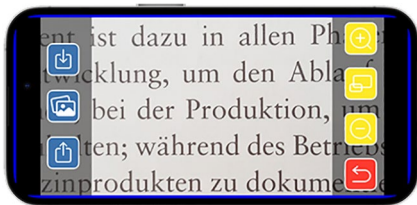EN

Zoom in

Enlarges the screen section

Overview function

Switches between the smallest and the previously set magnification


Zoom out

Reduces the screen section

Show / hide reading support

You can use the “Show and hide reading support” function to show reading support in the current screen section. You can activate and deactivate the reading support by pressing the  button.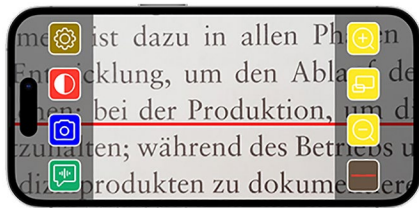

Troubleshooting

If the device no longer functions as described in these instructions, there may be a fault that you can rectify yourself.

There are four possible solutions that you can try:

1. Repeat the procedure that did not work again.
2. Press the RESET button B under the folding foot of the camera module. Then close and restart the app (as described here:

<https://support.apple.com/de-de/109359>). Repeat the procedure that did not work again.

3. Reset the camera module and the app to the factory settings. Then repeat the initial setup.
4. Delete and reinstall the app. Reset the camera module to factory settings. Then repeat the initial setup.

EN

If the above assistance does not solve your problem, please contact your specialist dealer for further support.

Function of the RESET button

Using the RESET button B only leads to a hard reset of the camera module. No settings are reset.

To reset the app and the camera module, proceed as follows and in this order:

1. Open the settings menu  in the app.
2. Reset the app using the “Reset to factory settings” function.
 - This function also resets the camera module if it is connected correctly.
3. If the camera module is not connected, the camera module must also be reset.

4. To do this, press and hold the On / Off button D until the status LED C flashes.
 - ✓ The components are reset.

Important information on resetting

During the initial setup, the factory default passwords are automatically overwritten to ensure the highest possible data security. From this point onwards, your camera module can only communicate with your app.

If one of the two components is reset, it will use the factory default password again. Communication with the other components is possible again after both components have been reset. Always reset both components together.

Care instructions

Clean the device with a soft, lint-free cloth. If the optical elements (camera, LEDs) are heavily soiled, e.g. with fingerprints or similar, lightly dampen the cloth.

- Avoid cleaning the device with:
 - Soap solutions that contain plasticisers
 - Alcoholic or organic solvents
 - Abrasive cleaning agents
 - Ultrasonic baths
 - Running water
- Failure to observe the instructions may result in damage to the device.

Disposal

This product is subject to the European WEEE directive on old electrical and electronic equipment in its currently valid version.

Do not dispose of your old device with household waste, but via an authorised waste disposal company or your municipal waste disposal facility.

Observe the regulations applicable in your country. If in doubt, contact your waste disposal organisation. Dispose of all packaging materials in an environmentally friendly manner.

Applicable standards

The device is a Class I medical device (non-sterile, without measuring function) and complies with Regulation 2017/745, Annex VIII, Chapter III, Rule 13. If the device is used outside the EU member states, the respective country-specific directives must be observed.

Hereby, Eschenbach declares that the radio equipment type (Art.-No. 16533) is in compliance with Directive 2014/53/EU.

The applicable declaration of conformity can be found on the following website.

<https://www.eschenbach-vision.com/de-DE/konformitaetserklaerungen>

UK Importer:

Associated Optical Products

13 Gore Road

SL18AA Burnham,Bucks

Internet: www.associatedoptical.com

Hereby, Eschenbach Optik GmbH declares that the radio equipment type optaro camera is in compliance with Radio Equipment Regulations 2017. The full text of the UK declaration of conformity is available at the following internet address:

<https://www.eschenbach-vision.com/de-DE/konformitaetserklaerungen>

Supplier's Declaration of Conformity**47 CFR § 2.1077 Compliance Information****Unique Identifier**

Product Name: optaro

Model No.: 16533

Responsible Party – U.S. Contact Information

Eschenbach Optik of America, Inc.

22 Shelter Rock Lane

Danbury, CT 06810

United States

Tel.: (800) 487-5389

Fax: (888) 799-7200

FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency

energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

MODIFICATION: Any changes or modifications not expressively approved by the grantee of this device could void the user's authority to operate the device.

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. End user must follow the specific operating instructions for satisfying RF exposure compliance. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

The portable device is designed to meet the requirements for exposure to radio waves established by the Federal Communications Commission (USA). These

requirements set a SAR limit of 4 W/kg averaged per ten gram of tissue. The highest SAR value reported under this standard during product certification for use when properly worn on the limbs.

Warranty

We guarantee the function of the product described in these instructions within the framework of the statutory provisions with regard to any occurring defects that are attributable to manufacturing faults or material defects. Damage caused by improper handling, including damage caused by dropping or impact, is not covered by the warranty. Warranty only by proof of purchase.

Trademarks

USB® is a registered trademark of USB Implementers Forum, Inc.

iPhone and App Store are trademarks of Apple Inc. registered in the USA and other countries and regions.

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- Tras conectarse correctamente, el LED de estado © del módulo de cámara cambia de azul a verde.
USB® es una marca registrada de USB Implementers Forum, Inc.
iPhone y App Store son marcas de Apple Inc., que están registradas en los EE. UU. y otros países y regiones.