



UNDERSTANDING YOUR RESULTS

Ovul will send your results to the app in the form of an image. The Ovul AI evaluates your sample for "ferning" crystals, which indicate the presence of estrogen in your saliva. The AI assesses the presence and density of these crystallization patterns and provides a percentage-based result, along with a color-coded prompt that frames the image result.

The percentage in the suggestions can be interpreted as how confident the AI is in its advice, how it interprets the image and relates it to the appropriate period of the cycle. **The final decision can only be made by the user or the doctor.**

That is, the answer regarding ovulation with a **90% probability** can be interpreted as: Artificial intelligence is almost (90%) sure that you are ovulating right now. **NOW IS YOUR TIME!**

HELPFUL TIPS

Ovul is an AI platform designed to learn and adapt to your unique rhythm. We HIGHLY recommend DAILY testing, even on days when ovulation is unlikely, as well as utilizing the Ovul App calendar. Consistent daily testing and accurate calendar tracking enable your Ovul device to recognize your personal fertility cycle with exceptional accuracy!

- ❖ Your answers to the registration survey will allow the Ovul AI to determine your ovulation more accurately.
- ❖ For best accuracy, apply a small amount of saliva to the center of the glass, avoiding bubbles.
- ❖ During testing, keep the Ovul device in a dry area. Excessive ambient moisture and humidity may affect the accuracy of your results. Store the tracker in a dry place.
- ❖ Do not open the Ovul device until the test is complete and results have been sent to the app.
- ❖ Clean the glass surface BEFORE each and every use to ensure accurate readings.
- ❖ Use video instructions from our Youtube channel - <https://youtube.com/@ovulai>

TAKE YOUR FIRST TEST!

- ❖ For best accuracy, NO SMOKING, FOOD, DRINKS, or MOUTHWASH 30 minutes prior to testing.
- ❖ To take a test, just open the Ovul tracker, it will signal its "Ready" status with a color light which also displays the device's charge level.
- ❖ Make sure the round black glass receptacle surface located in the center of the lower portion of the Ovul device is clean and free of any particles.
- ❖ Use a clean fingertip to collect a small amount of saliva from under your tongue.
- ❖ Smear a THIN LAYER of saliva onto the black glass receptacle. Close the device.
- ❖ The Ovul device will blink white to let you know that it's starting to analyze.
- ❖ Place the device on a stable, flat surface during testing to prevent it from falling or getting damaged.
- ❖ You will see a timer in the Ovul App indicating that your results are on their way. You will receive your results directly to the Ovul App once the analysis is complete.
- ❖ Repeat as needed!

FAQ

What should I do if the device does not flash blue or does not display any light indication when opened?

Charge the device and try again.

How often should I charge the device?

The device displays the charge level with a color indication when you open it. If the device flashes red or a notification appears in the application, charge the device.

How long does it take to charge and how do I know if the device is fully charged?

A full charge takes about 6 hours, when the device is fully charged, the red indication will change to green.

What should I do if my device flashes blue after opening?

You need to reconnect the device to your home Wi-Fi network. To do this, use the "Reconnect Ovul" button on the app settings page.

How do I clean the device?

Wipe the glass surface with the soft cloth provided. Do not use harsh chemicals. Wipe the glass surface before each use.

How often should I test?

Testing daily and regularly maintaining the Ovul App Calendar will greatly improve Ovul AI's accuracy and your chances of success!

What do I do if I accidentally open my Ovul device during an on-going test?

Not a problem! Just close the device and it will start a new test of your saliva sample.

When is the best time to test for ovulation?

We recommend testing daily for the first 2 months. This will allow our AI to best understand your personal cycle and catch your ovulation window at its earliest stage (usually 4 days prior)

How do I know if I'm ovulating?

Ovul recognizes changes in estrogen levels based on a saliva sample. Our AI will help you categorize the result and provide suggestions on the percentage of confidence, as well as a visual color-coded prompt.

How early can Ovul see my ovulation window?

When testing daily in the week leading up to your period, Ovul can predict the start of ovulation up to 4 days in advance.

Based on my Ovul result, when is the best time for intercourse?

Numerous studies show that the best time to get pregnant is in the beginning of the fertile window. Ovulation occurs after that, the chances of getting pregnant decrease.

What do the percentages in my Ovul result mean?

Our AI Assistant will deliver results based on percentages. Think of it as a "chance of rain" but more reliable! Please review the "Understanding My Results" Section of these instructions.

TO GET STARTED, YOU NEED:

- 1.** change your ovul device before use
 - 2.** download the ovul mobile application
 - 3.** connect your ovul to the Wi-Fi network
 - 4.** start testing!
- use the QR code



Ovul Saliva fertility tracker V2.2

DOWNLOAD AND REGISTER WITH THE OVUL APP

- ❖ Using your smartphone, download the "Ovul" app from the App Store or use the QR code on the back of these instructions for Android devices.
- ❖ Open the Ovul App and complete the registration survey. After the survey, you will be prompted to connect the device.
- ❖ Select "Connect" and follow the instructions to pair the tracker and mobile app

CONNECT YOUR OVUL FERTILITY TRACKER

- ❖ Before proceeding, make sure your device is fully charged and closed.
- ❖ Open the Ovul App on your phone.
- ❖ Open the Ovul device by separating the top and bottom, and wait until it starts flashing blue. If it does not, close the device, wait a few seconds and open the device again.
- ❖ Follow the instructions in the app to connect the Ovul tracker to your home Wi-Fi network and pair it with the mobile application.

FCC ID: ZBH4A-C00301

FCC Compliance Statements

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

For more Compliance Statements, please refer to the full version of Product Description.