Thank you for choosing Lumos Glasses. We want you to get the most out of your experience. Please read this leaflet and the accompanying documentation carefully before using your glasses, mobile app, Lumos Services and any other related services. Failure to follow the instructions below could result in product damage, malfunction or even injury.

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How Lumos Glasses work

LED sources emit targeted wavelengths of light onto the lenses, which have special coatings that reflect these wavelengths back into your eyes. In doing so, the glasses mimic the healthy effects of the outdoor sky on our biology. When the LEDs are turned off, the coating serves as a blue light filter, blocking incoming blue light and mimicking the night time sky.

Intended use

- **Purpose** Light therapy to support sleep, mood, and circadian rhythm regulation.
- **User group** Adults 18+. Teens 13–17 may use it under adult supervision.
- **Disclaimer** Not intended to diagnose, treat, or prevent any medical condition.

Safety information

Before using Lumos Glasses, consult a healthcare professional if you:

- Have a history of photosensitive epilepsy, seizure disorders, or photophobia.
- Have been diagnosed with diabetic retinopathy, ocular migraines, or other serious eye conditions.
- Are taking photosensitizing medications, such as certain antibiotics, diuretics, or antipsychotics.
- Are pregnant, under 12 years of age, or recovering from recent eye surgery.

Discontinue use and consult a healthcare provider if you experience headaches, eye strain, dizziness, or visual disturbances during use.

Warnings

- This device emits bright light. Do not shine the light directly into other people's eyes.
- Do not use while driving, operating machinery, or performing tasks that require full attention.
- Keep the device out of reach of children.
- If unsure about suitability, consult a healthcare professional before use.

Light exposure & timing

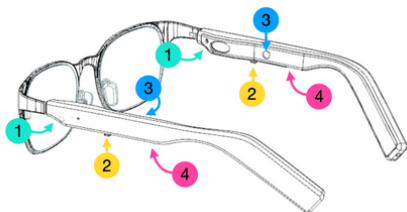
- Use the device during daytime hours, typically between 5:30 AM and 5:30 PM, as guided by the app.
- Avoid evening or nighttime use unless directed by a healthcare provider or specific app-based protocol.
- Future versions of the Lumos app may support clinically validated programs for jet lag, shift work, and other non-standard use cases.

Care & handling

- **Keep dry:** Lumos Glasses are not waterproof. Avoid exposure to water, rain, or high humidity.
- **Clean gently:** Use only the provided dry microfiber cloth. Do not use solvents, alcohol, or abrasive cleaners.
- **Charge safely:** Use only the included USB cable. Ensure all components are completely dry before charging.
- **Store properly:** Store at room temperature. If unused, charge the battery to 50% every 3 months.
- **Protect from damage:** Avoid extreme heat, cold, or direct impact.
- **No user servicing:** Do not attempt to disassemble, modify, or repair the device yourself.

Hardware information

Here are the components of Lumos Glasses hardware:

1. **Ambient light sensors** These light sensors are calibrated to measure your environment melanopic lux levels, which corresponds to how much light that activates your circadian rhythm. We generally want a high amount of melanopic lux during the day and none at night time. To view this data, you must download Lumos Glasses mobile app
2. **Mechanical buttons** Short press on the button toggles the main therapy light for each arm. Long hold for 3 seconds then letting go toggles the Bluetooth mode for connecting with the mobile device. It is also used for a reaction time game to test your cognition.
3. **Proximity sensors** These sensors, one on each arm, is used to detect if you are wearing the glasses or not by the mobile device, which can help track your progress and optimize your wellness.
4. **Charging ports** The charging ports use micro-USB connections. Any generic micro-USB cable should charge them. And a USB to dual micro-USB Y cable is provided to charge both at once from a single USB port.

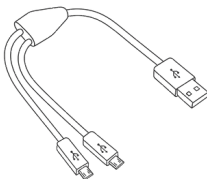
Setup instructions

To get started using Lumos, follow these simple instructions.

STEP 1: Charge your glasses

Start by charging your Lumos Glasses to full charge. To do this:

1. Plug the USB end of the charging cord to any usb port (ex. laptop or an outlet adapter)
2. Plug the other two (2) micro-USB charging cords into each leg of the glasses.
3. The glasses will have an orange glow near each charging port to indicate it is charging, and the orange glow will stop once charging is completed.

STEP 2: Get the mobile app

Scan the QR code below to download the latest version of the Lumos App on your mobile device. You can also visit

www.lumosglasses.com/app

STEP 3: Turn on Bluetooth

1. Hold the mechanical buttons down for 3 seconds until the light starts blinking and then let go. Do this for both sides.
2. Your glasses are now ready for connecting to a mobile device.

If this doesn't work, the glasses may be out of battery. See the section in this leaflet on "Charging the glasses".

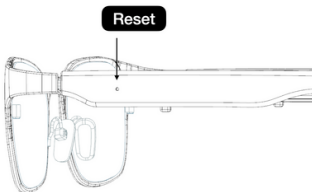
This device uses Bluetooth Low Energy (BLE) operating in the 2.4 GHz ISM band.

STEP 4: Connect Lumos Glasses to your mobile device

1. Open the Lumos app on your device.
2. Make sure to grant the app permissions
3. Ensure that the glasses are in Bluetooth connection mode (blinking).
4. Once the app is booted up, it will search for the glasses and make a connection.
5. Once the glasses are connected, the lights will stop blinking, and the app will indicate that it is connected.

Resetting your Lumos Glasses

If for any reason the glasses are not functioning correctly even though they are charged (e.g. not connecting with Bluetooth) you can reset the glasses by using the reset pinhole. Insert a paperclip or SIM card pin into the hole for a brief second, and the device will reset and the lights will begin flashing. If a mobile device with the Lumos app is present, the glasses should automatically reconnect.

Specifications

Frame & fit

- Sizes: 129mm / 135mm / 142mm
- Front: Stainless steel, matte black finish
- Temples: TR-90, lightweight & flexible
- Nose Pads: Adjustable silicone
- Hinges: Spring-loaded
- Weight: 36.3g (without lenses)

Lenses

- Optical glass, 16.3g
- Scratch-resistant, wavelength-selective coating
- Unisex rounded rectangular shape

Light therapy engine

- Wavelength: 470–480nm
- Brightness: 550 melanopic lux
- Beam aimed at lower nasal retina
- Recommended: 20+ minutes daily

Power & battery

- Battery Life: Up to 10 hours
- Charging Time: ~60 minutes
- Charging: Dual micro-USB
- Battery: Lithium polymer

Connectivity & Controls

- iOS/Android app (Bluetooth Low Energy)
- On-frame button & app-based control
- App: Schedule sessions, track usage, adjust brightness

Sensors & intelligence

- Ambient light sensor (melanopic lux)
- Wear detection: auto on/off

Environmental

- Not waterproof — avoid washing
- Charging Temp: 0°C to 45°C
- Use Temperature: -10°C to 60°C
- Store at 25°C ± 5°C, 65% ± 20% RH
- Charge to 50% every 3 months during storage

Intellectual property rights

Lumos™, product names (such as *Citizen*™), the Lumos logo, and the design of the glasses are trademarks of Lumos Health Inc. Aspects of the product design and technology are protected by copyright and trade secret laws and are patented or patent-pending. All rights reserved. Unauthorized use or reproduction is prohibited.

Terms of use & limitations of liability

Use only as intended. Lumos Health Inc. is not liable for any misuse, unauthorized modification, or failure to follow usage guidelines. The device is not a substitute for medical treatment. Use of the product and app constitutes acceptance of our full terms at **www.lumosglasses.com/legal**.

Contact information & support

For product questions, technical support, or returns, please contact:

Lumos Health Inc.
544 Brookhaven Crescent
Waterloo, Ontario, Canada N2L 4R7

 www.lumosglasses.com
 hello@lumosglasses.com

Warranty & returns

Lumos Glasses are covered by a 1-year limited warranty against manufacturing defects. This warranty does not cover damage due to misuse, water exposure, or unauthorized repairs.

To initiate a return or warranty claim, please retain your box serial number and contact our support team.

Manual version

- Document Version: v1.0
- Release Date: May 2025
- Applies to: Lumos Glasses – Consumer Edition

© 2025 Lumos Health Inc. All rights reserved.

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Regulatory and Certifications

English

Lumos Glasses are a Class I medical device.

Canada: Complies with Health Canada Medical Devices Regulations (SOR/98-282); manufacturer holds a valid Medical Device Establishment Licence (MDEL).

USA: Listed under FDA 21 CFR 880.5700 as a general wellness device. Exempt from 510(k) premarket notification.

This product is not intended to diagnose, treat, cure, or prevent any medical condition.

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Photobiological Safety

English

Lumos Glasses were evaluated under IEC 62471:2006 by TÜV SÜD. At the standard 200 mm distance, the device is classified as Exempt Group, indicating no photobiological hazard under typical use conditions.

While IEC 62471 does not classify exposure at closer distances, Lumos commissioned additional near-eye measurements, performed by TÜV SÜD. These evaluations incorporated eye behavior modeling, optical simulations, and Lumos' custom calibration process. The findings confirmed that near-eye exposure remains well below accepted thresholds, with a >10× safety margin.

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Battery Transport Safety

English

The built-in rechargeable lithium-polymer battery complies with UN 38.3 testing standards for safe air and ground transportation.

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Electronic Waste Compliance

English

Do not dispose of this product with household waste. This product contains electronic components and a rechargeable battery and must be recycled at an appropriate collection facility in accordance with local regulations. Complies with the WEEE Directive 2012/19/EU.

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Disposal Instructions

English

In Canada and the United States, dispose of the device through authorized e-waste recycling programs or contact Lumos Health Inc. for return instructions.

In the EU or UK, recycle the product through official WEEE collection points.

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Battery Considerations

English

Do not attempt to remove or replace the battery unless instructed by Lumos Health Inc. If the device is returned for servicing or disposal, our team will handle the battery according to regional environmental standards.

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FCC & ISED Wireless Compliance

FCC Compliance Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. this device may not cause harmful interference, and
2. this device must accept any interference received, including interference that may cause undesired operation.

CAUTION: The grantee is not responsible for any changes or modifications not expressly approved by the party responsible for compliance. Such modifications could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

FCC & ISED Wireless Compliance

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception — which can be determined by turning the equipment off and on — the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This equipment has been tested and meets applicable limits for radio frequency (RF) exposure.

Canadian Compliance Statement (ISED)

This device contains license-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada license-exempt RSS(s).

Operation is subject to the following two conditions:

1. This device may not cause interference.
2. This device must accept any interference, including interference that may cause undesired operation of the device.

NOTE: This equipment has been tested and meets the applicable limits for radio frequency (RF) exposure under RSS-102.

English

GTIN: 990312280491

Each pair of Lumos Glasses includes a unique serial number. The GTIN and box serial number are printed on the packaging and linked to internal component identifiers. These are used for traceability, warranty validation, batch tracking, and recall readiness.

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