

WRIST TRANSMITTER (WTD-1)

The Wrist Transmitter is used to monitor the wellness of a subject who is home alone.

● BATTERY

The WTDI uses one 3V 500mAh lithium battery and will last for at least 12 months

● GETTING STARTED

- Wear the WTDI on the subject and secure the wristband.
- Press the button and hold for 15 minutes. The WTDI is now in Normal Operation Mode.
- Under Normal Operation Mode, WTDI transmits a functional signal to the Control Unit every 30 seconds.
- When the wristband has been broken, WTDI sends a dysfunctional signal to the Control Unit and goes to Reset Mode.
- After a 2-day Reset Period, WTDI is again ready for activation by following point #1 of above.

This device complies with Part 15 of the FCC Rules
Operation is subject to the following two conditions:
(1) This device may not cause harmful interference, and
(2) This device must accept any interference received,
including interference that may cause undesired operation

CAUTION: Any changes of modifications not expressly approved by the grantee of this device could void the users authority to operate the equipment.