


DO NOT OPEN  
UNTIL READY FOR USE

ACTIVATE Capsule before swallowing

1. Align lens end of Capsule with Activation Port on right side of Monitor.
2. Press Activate Sensor key on front of Monitor while holding lens end of Capsule in Activation Port.
3. Follow prompts on Monitor display.
4. If activation fails, try to activate again after 5 seconds, by repeating the first 3 steps above.
5. When activation is successful, swallow Capsule with water.

CAUTION: DO NOT CHEW CAPSULE  
DO NOT INGEST CAPSULE IF  
VISIBLY DAMAGED!

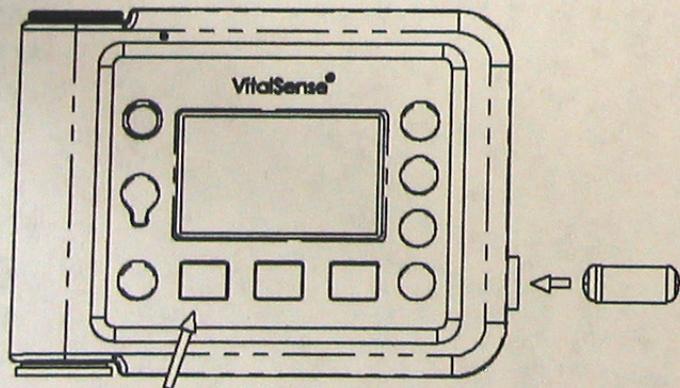

DO NOT OPEN  
UNTIL READY FOR USE

How Supplied: 1 Capsule per package.

Single-use Capsule.

Reception Range: 1 meter

Operating Time: 240 hours

Mini Mitter Company, Inc.  
20300 Empire Ave., Bldg. B-3  
Bend, OR 97701 USA  
541-322-7272  
www.minimitter.com

FCC ID#: JIAXTP1

SERIAL #

EXPIRATION DATE

