

Wireless data transmission principle: Wireless data transmission function of the oximeter is controlled by the software "Medview" on the PC.

Data transmission process:

Firstly, Oximeter setting: set the bluetooth mode as on by menu selection.

Secondly, PC Setting: plug the bluetooth adapter on the computer and than Start the Medview" software, select "looking for matching devices" to build the connection with the oximeter. The connection will be built within 10 seconds. and then select "data transmitting" menu to get data from the oximeter, and select "stop "menu to stop wireless communication with the oximeter, and the communication will be stopped within 1 seconds.