

#### [Bluetooth pairing]

- Bluetooth pairing with Nokia 2720.
- [BA-100] Press the button for more than 3 seconds.
- [BA-100] Confirm that the red LED is ON.
- [Nokia 2720] Go to “Bluetooth menu → Paired devices.” Select “Add new device” for searching for the Bluetooth adapter.
- [Nokia 2720] Find a device name, “AMC\_JandJ”
- [Nokia 2720] “Add” AMC\_JandJ for pairing.
- [Nokia 2710] Input “1234” for the passcode.
- [BA-100] After a while, the green LED turns ON.
- [BA-100] After a while, the red LED turns ON.
- [BA-100] And then, the green and red LEDs turn OFF.
- [Nokia 2720] Run MobiCliniQ software
- [BA-100] Connect a glucometer to the Bluetooth adapter and press the button for test.
- [Glucometer] “PC” is shown.
- [BA-100] The red LED blinks twice. The green LED blinks for a while. And then, the red and green LEDs turn OFF.
- [Nokia 2720] See the messages on MobiCliniQ software

[Manual data transmission]

- Press the button of the Bluetooth adapter while a glucometer and the adapter are being connected. The whole readings stored in the glucometer will be automatically downloaded into the Nokia 2720 and be shown in MobiCliniQ software.